SAM®

AXIOSPLIT®

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*ZERO Based
Reference
Articulator System
with Magnetic
Mounting Plate*

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AXIOSPLIT® NORM AND MAGNETIC SPLITCAST SYSTEM

 *The AXIOSPLIT® System synchronizes two or more SAM® Articulators such that mounted casts can be transferred interchangeably without any clinical errors. By having one Axiosplit articulator in the laboratory and one in the office you need only send your mounted casts.*

 *Accuracy checking can be done with original master checking gauge ASP 470 or with the split-cast verification device ART 396.*

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BENEFITS OF THE AXIOSPLIT[®] SYSTEM

*Identical articulators in dental laboratory
and dental office!*

*Quick, easy, and precise removal
and reattachment of casts!*

*Easy and quick exchange of magnets with
different magnetic strength!*

*Matrix holders optimized for use
with all gypsum types!*

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AXIOSPLIT® Magnet Holding Cups are removable by loosening the Thumb Screw. Magnets with different holding force can be easily interchanged to meet your needs.

ASP 580 AXIOSPLIT® Magnet N

ASP 581 AXIOSPLIT® Strong Magnet S

SYNCHRONIZING ARTICULATORS

Use the appropriate AXIOSPLIT® Starter Kit identified below for the first articulator:

- SAM® 2P, 2PX or SAM® 3::
- For each additional articulator

ASP 509K

AXIOSPLIT® Articulator KIT
ASP 514K

- GC-Pattern Resin, a plastic mixing cup and a spatula

- 1** Set Condylar Housing Inclination to 45° and Bennett Guidance Angle to 5° with the straight Bennett inserts. Examine the adjustment of your articulator to be sure condylar element contacts posterior housing wall and Bennett Guidance Inserts. If necessary you have to adjust the articulator again
- 2** Clean the mounting plate bearing surfaces of upper and lower part of your articulator.
- 3** To attach the one piece ASP Primary Plate to the Upper Member of the Articulator, place ASP Plate onto mounting plate pins. While holding ASP Plate with moderate clockwise pressure, tighten Thumb Screw firmly.
- 4** Fasten the master incisal pin with the knurled thumb screw of the incisal table in the lower part of the articulator. Flat surface of incisal pin should be positioned to touch the underside of the upper member of the articulator.
- 5** Fasten the brightly anodized screw base of the adjusting plate to the articulator lower part in the same manner as described in 3 above.

- 6** Place upper member of articulator upside down on table top. Attach Master Checking Gauge to AXIOSPLIT[®] Plate on upper member of articulator with angulation towards condylar housings.
- 7** Loosen centric locking devices on your articulator. Attach hollow AXIOSPLIT[®] plate to lower side of Master Checking Gauge.
- 8** Carefully place lower member of articulator on the upper member. Check to see that only incisal pin and condylar elements are in contact. Remove lower member of articulator.
- 9** Place 20 drops of GC Monomer in plastic mixing bowl. Add enough GC Resin Powder to make a highly fluid mix of resin. Place mixed resin in 3 areas on the inner wall of the hollow AXIOSPLIT[®] plate.

- 10** Insert articulator lower member into upper member in a straight vertical direction until master incisal pin and condylar elements are fully seated. First, tighten left centric locking screw and then the right one.
- 11** After resin has hardened. Invert articulator and loosen centric locking screws or devices. Hold Master Checking Gauge against lower member of articulator and remove articulator upper member.

- 12** Remove AXIOSPLIT® Magnet from upper AXIOSPLIT® Plate..
- 13** Place upper member of articulator back on lower member and lock the centric. Check fit of AXIOSPLIT® Plates with SHIM STOCK. Normally, it is OK. If not, unscrew the AXIOSPLIT® Plate containing the resin.

Remove magnet. Heat mounting plate with a flame to soften plastic. Clean and remove plastic and repeat steps 8 to 15.

- 14** If procedure has to be redone, then repeat steps 11 to 13.
- 15** Attach AXIOSPLIT® Security Rings ASP 435 to upper and lower thumb screws with flat edge against articulator surface. The security rings prevent accidentally unscrewing of AXIOSPLIT® Plates. Label articulator with Master Gauge Serial Number.

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cast mounting device with quick removable thumb screw/magnet assembly for comparison with following articulators SAM® 2P, SAM® 2PX, and SAM® 3, master check and master incisal pin are not included

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MOUNTING PLATES FOR AXIOSPLIT®, OVAL

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(20 pieces) yellow preformed precision matrix mounting plates, plastic, with extended posterior and lateral edges and metal insert, eliminates the need to make a gypsum matrix, for single-use only

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MOUNTING PLATES FOR AXIOSPLIT® , OVAL BP

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(100 pieces) bulk pack, yellow preformed precision matrix mounting plates, plastic, with extended posterior and lateral edges and metal insert, eliminates the need to make a gypsum matrix, for single-use only

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MOUNTING PLATES FOR AXIOSPLIT® , ROUND

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MOUNTING PLATES FOR AXIOSPLIT® , ROUND BP

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(two pieces) metal, used to cover the mounting plate thumb screw to prevent accidental unscrewing of the attached AXIOSPLIT® plates

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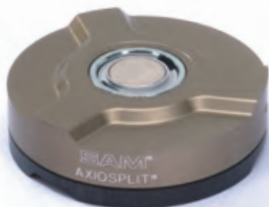
+ 15 mm, used to attach SAM[®] 2C AXIOSPLIT[®] mounted casts on a SAM[®] 2P, SAM[®] 2PX or SAM[®] 3 articulator equipped with the AXIOSPLIT[®] system. Complete assembly, includes the AXIOSPLIT[®] magnet.

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for the production of secondary matrix holder made of gypsum with rubber rings and lubricant liquid

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*with master incisal pin
for SAM® 2P, SAM® 2PX, and SAM® 3*

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The proven modelling resin for making all kinds of custom attachments, fixations, implant abutments or resin dies for electroforming technique.

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includes master check with incisal pin (ASP 470), regular matrix holder (ASP 529), matrix holder, two-piece assembly (ASP 539) each with magnets with normal strength (ASP 580), security rings (ASP 435), and 20 oval mounting plates (ASP 420) for SAM[®] 2P, SAM[®] 2PX, and SAM[®] 3

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includes master check with incisal pin (ASP 470), regular master holder (ASP 530), master holder, two-piece assembly (ASP 540) each with strong magnets (ASP 582), security rings (ASP 435), and 20 oval mounting plates (ASP 420) for SAM[®] 2P, SAM[®] 2PX, and SAM[®] 3,

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for every other articulator with master check system, includes regular matrix holder (ASP 529), matrix holder, two-piece assembly (ASP 539) each with magnets with normal strength (ASP 580), and security rings (ASP 435)

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for every other articulator with master check system, includes regular matrix holder (ASP 530), matrix holder two-piece assembly (ASP 540) each with strong magnets (ASP 582), and security rings (ASP 435)

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*screw type metal plate for articula-
tor upper member, threaded and
supplied with magnet with normal
strength*

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*screw type metal plate for articulator upper
member, threaded and supplied with
strong magnet*

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ASP 539

AXIOSPLIT® MATRIX HOLDER N, TWO-PIECE ASSEMBLY

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*for adhesive adjustment in the
lower part of articulator,
screw type metal plate
supplied with magnet with
normal strength*

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ASP 540

AXIOSPLIT® MATRIX HOLDER S, TWO-PIECE ASSEMBLY

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*for adhesive adjustment in the
lower part of articulator, screw
type metal plate supplied with
strong magnet*

AXIOSPLIT[®] MAGNET IN WECHSELTOPF

ASP 580

AXIOSPLIT[®] MAGNET N WITH MAGNETIC CUP

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*AXIOSPLIT[®] magnet N with magnetic cup
with magnet with normal strength*

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*AXIOSPLIT[®] magnet S with magnetic cup
with magnet with strong strength*

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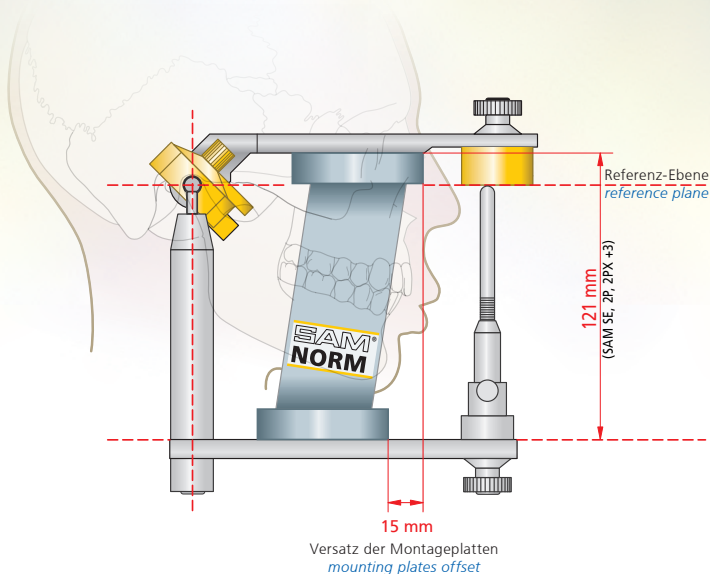
*adjustable cast holder
includes strong magnet*

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The SAM[®] Norm, in combination with the anatomic facebow, assures the correct positioning of the upper cast in the articulator. The aesthetics and function are accurately simulated for Orthodontics and Prosthodontics.

SAM[®] • AXIOGRAPH[®] • AXIOSPLIT[®] • AXIOQUICK[®]
AXIOSAW[®] • AXIOPIN[®] • AXIODRILL[®]
AXIOWAX[®] • AXIOTRON[®]

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*are registered trade marks of the
SAM[®] Präzisionstechnik GmbH*

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