SAM®

AXIOQUICK®

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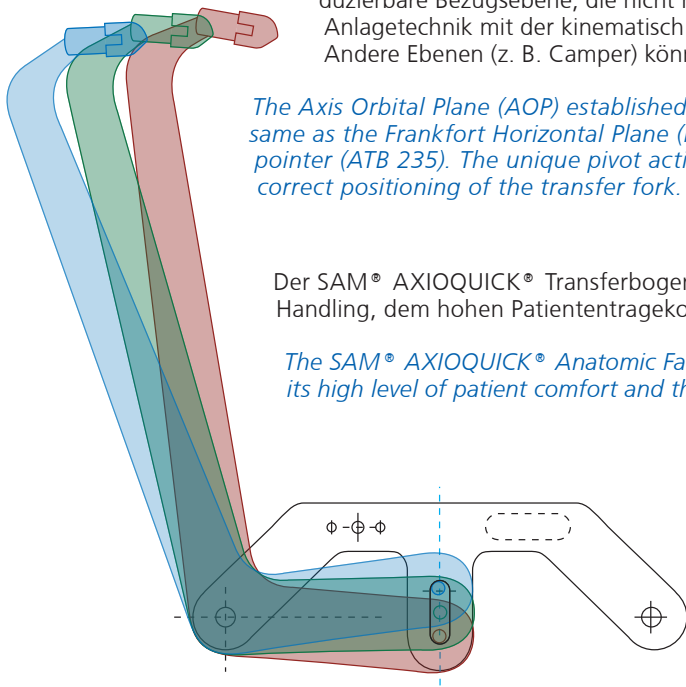
Articulator mounted casts that are correctly related to the cranium are essential part of diagnosis and therapy. An anatomically related maxillary cast when used along with a correct centric record should be used with customized condylar pathway settings or even with average values.

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The Axis Orbital Plane (AOP) established by using Anatomic Porion-Orbitale reference plane is the same as the Frankfort Horizontal Plane (FHP). Other planes maybe selected using the reference pointer (ATB 235). The unique pivot action of the SAM® facebow guarantees the optimum and correct positioning of the transfer fork.

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The SAM® AXIOQUICK® Anatomic Facebow is impressive about its light weight, its easy handling, its high level of patient comfort and the easy application technique.

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The FACEBOW for Esthetics and Function

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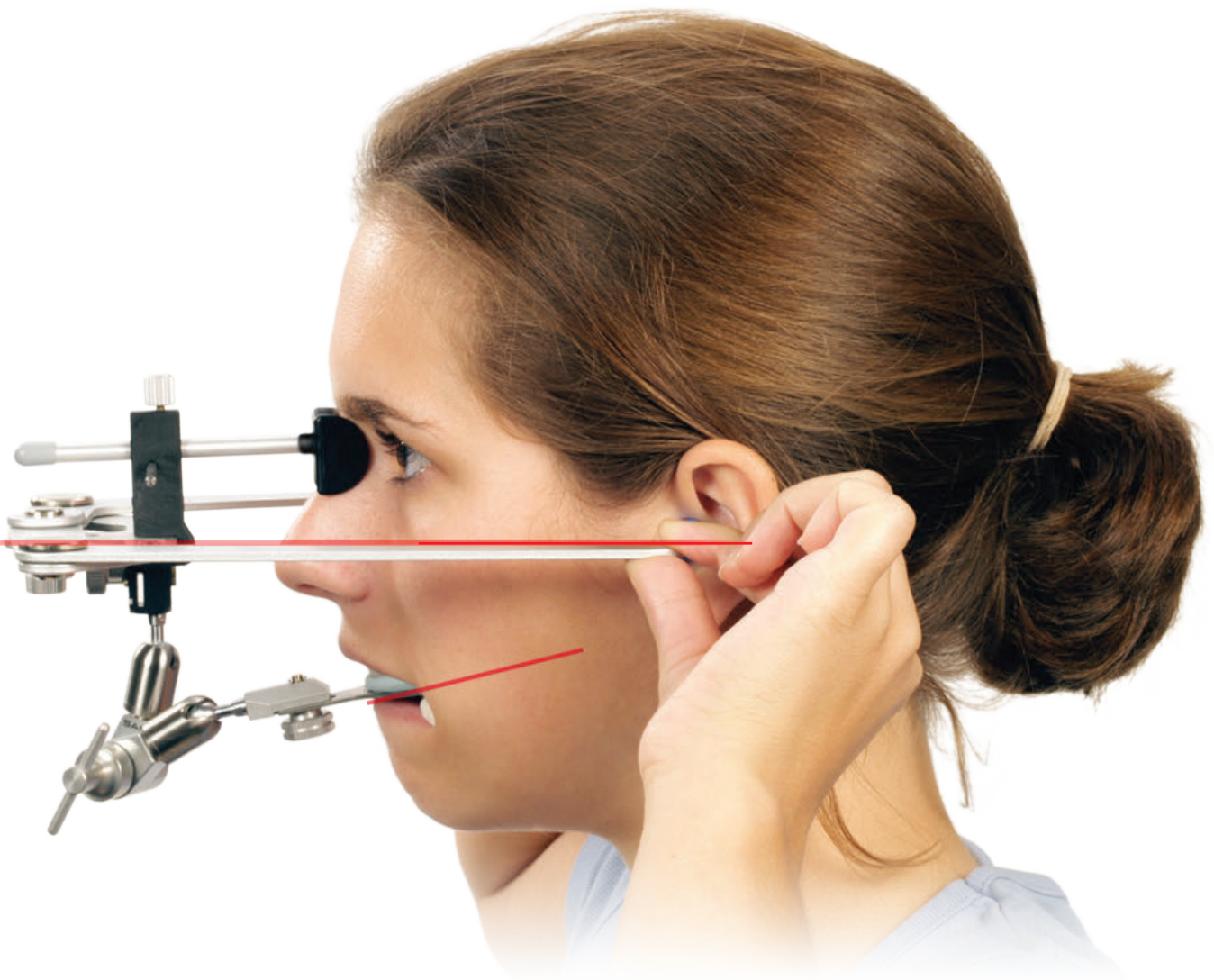
The special geometry and mechanics of the SAM® facebow provides an automatic cranial related symmetry. The Anatomic Porion -Orbitale oriented and cranial related attaching technique assures that the cranium and maxillary teeth esthetics are correctly transferred to the articulator.

The single use blue hygienic earpiece caps for single use aid in the correct positioning in the ear canal, as well as the wearing comfort for the patient.

The patented nasion relator with the interpupillary line leveler helps to assure the parallel relationship between the interpupillary line and the facebow. The silicone nasion support makes the transfer more comfortable for the patient.

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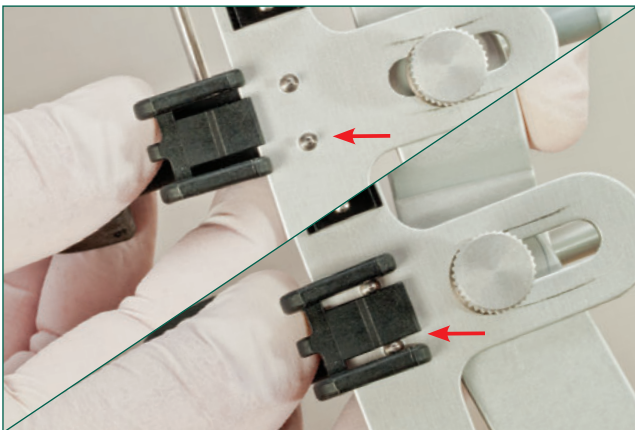
The blue hygienic earpiece caps (ATB240) are placed on to the black earpieces of the facebow.

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
The notch is pointing outwards (1), the flat angled-side facing inward (2).

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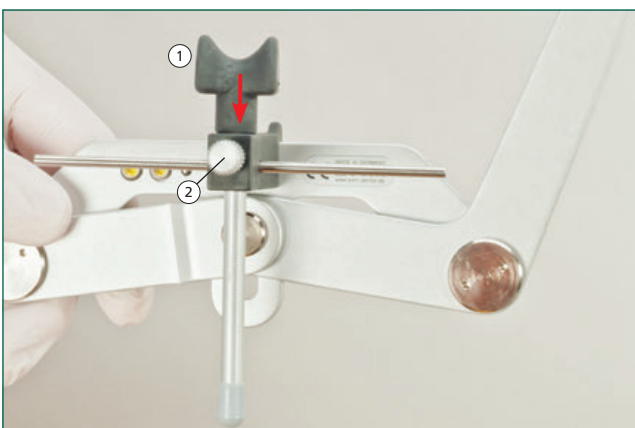The nasion relator (ATB 394) is properly positioned on the facebow (ATB 303) when it is fully inserted between the pins on the underside of the facebow.

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The nasion support should be fully „retracted“ (1) and lightly held in place with the screw (2).

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
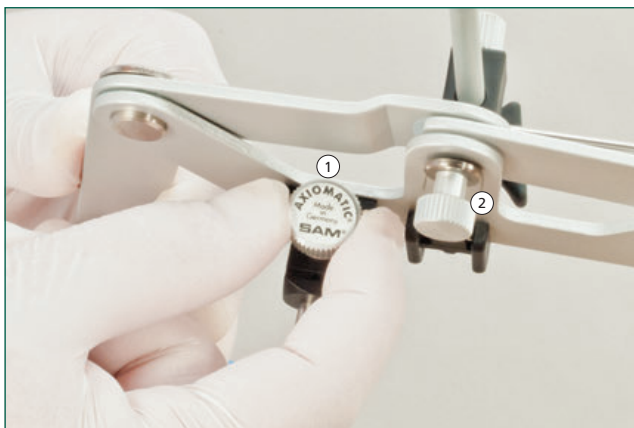
The transfer fork assembly is pushed onto the sliding attachment at the bottom of the AXIOQUICK® face bow until fully seated.

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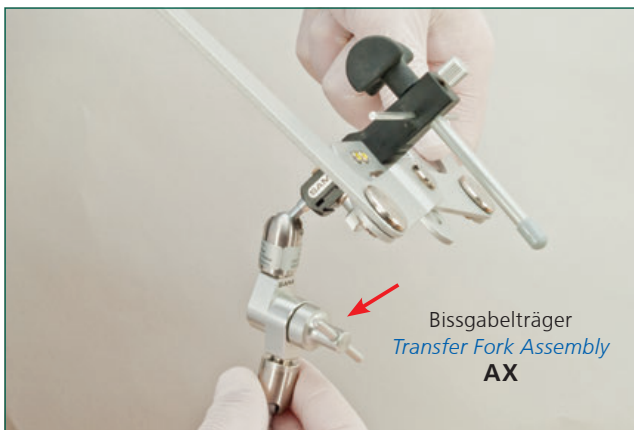
Tighten the knurled thumb screw on the transfer fork assembly (1). The thumbscrew on the bottom of the facebow (2) should be loosened, so that the side arms of the facebow are freely moveable.

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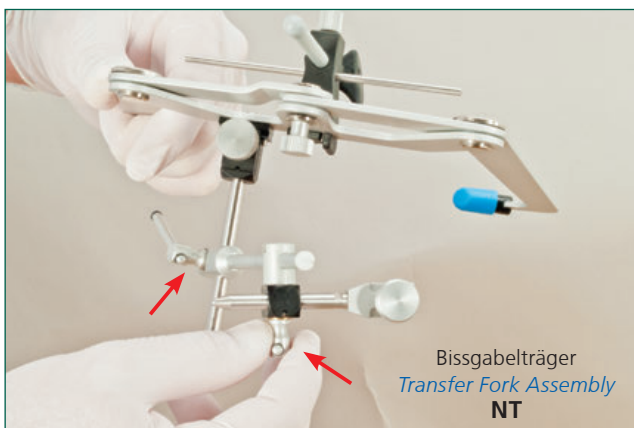
The AX bar clamp or NT bar clamps (red arrow) should be loosened only by half a turn.

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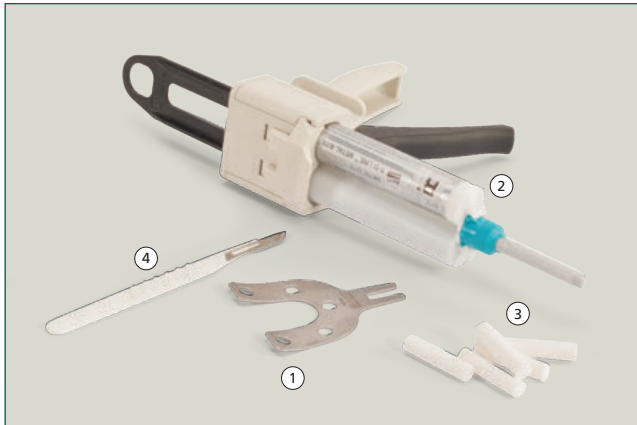Alternatively you could use Bite-Tabs (Panadent®) or other materials.

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Place an even thin layer of PVC material on the top surface of the transfer fork.

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The transfer fork is positioned as shown here with projections in the midline. The transfer fork is held without movement until PVS is set.

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After hardening the transfer fork is removed and all deep impressions, molded fissures and interdental spaces removed.

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The transferfork is reseat on the maxillary teeth and held in place by having patient biting on cotton rolls.

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The face bow is held with thumb and index finger only on the corner of the earpiece by the patient.

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The patient slowly inserts the blue capped earpieces into the external auditory canals until he hears worse.

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Now the patient push gently with the the thumbs the earpieces upwards and then directly inwards to seat in the small superior depression in the ear canal. At this point the patient can hear. If not, earpiece procedure is repeated.

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
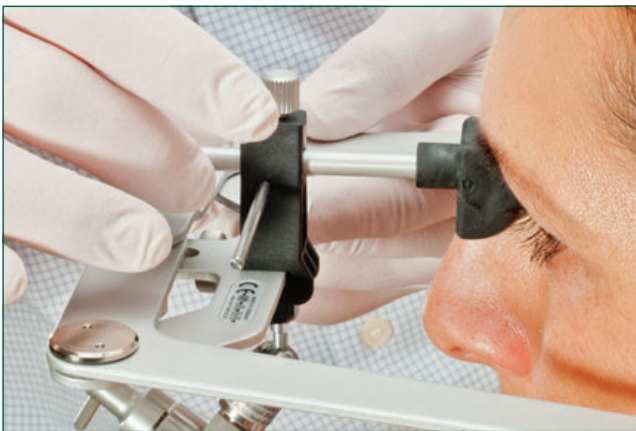
*When positioned correctly, there should be space (1) seen below the earpieces and the patient can hear. It is very **IMPORTANT** that he hold the earpieces in the correct position while the operator positions and check the interpupillary line leveler.*

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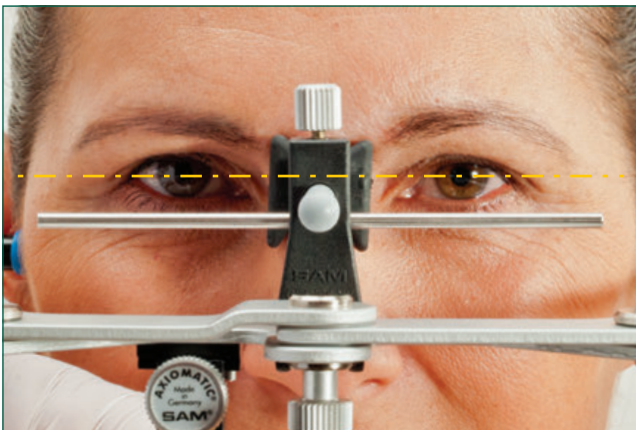The nasion relator is in light contact with nasion and the interpupillary line leveler is checked for proper positioning.

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As seen here, the yellow line on the interpupillary plane, identifies a parallel face bow esthetically correct anterior reference plane related to orbitale anteriorly and the posterior anatomic porion reference points.

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After proper position is checked, and patient is still holding earpiece end of face bow, the knurled thumb screw is tightened.

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Connecting the transfer fork assembly AX

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With the toggle of the transfer fork assembly is half open, the AX transfer fork can be easily moved into position to be attached to the face bow transfer fork.

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Tip: The handling of the transfer fork assembly is much easier if the set screw is facing down

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With one hand push the shoe of the transfer fork assembly complete on the conical groove of the transfer fork assembly . The other hand secures the joint.

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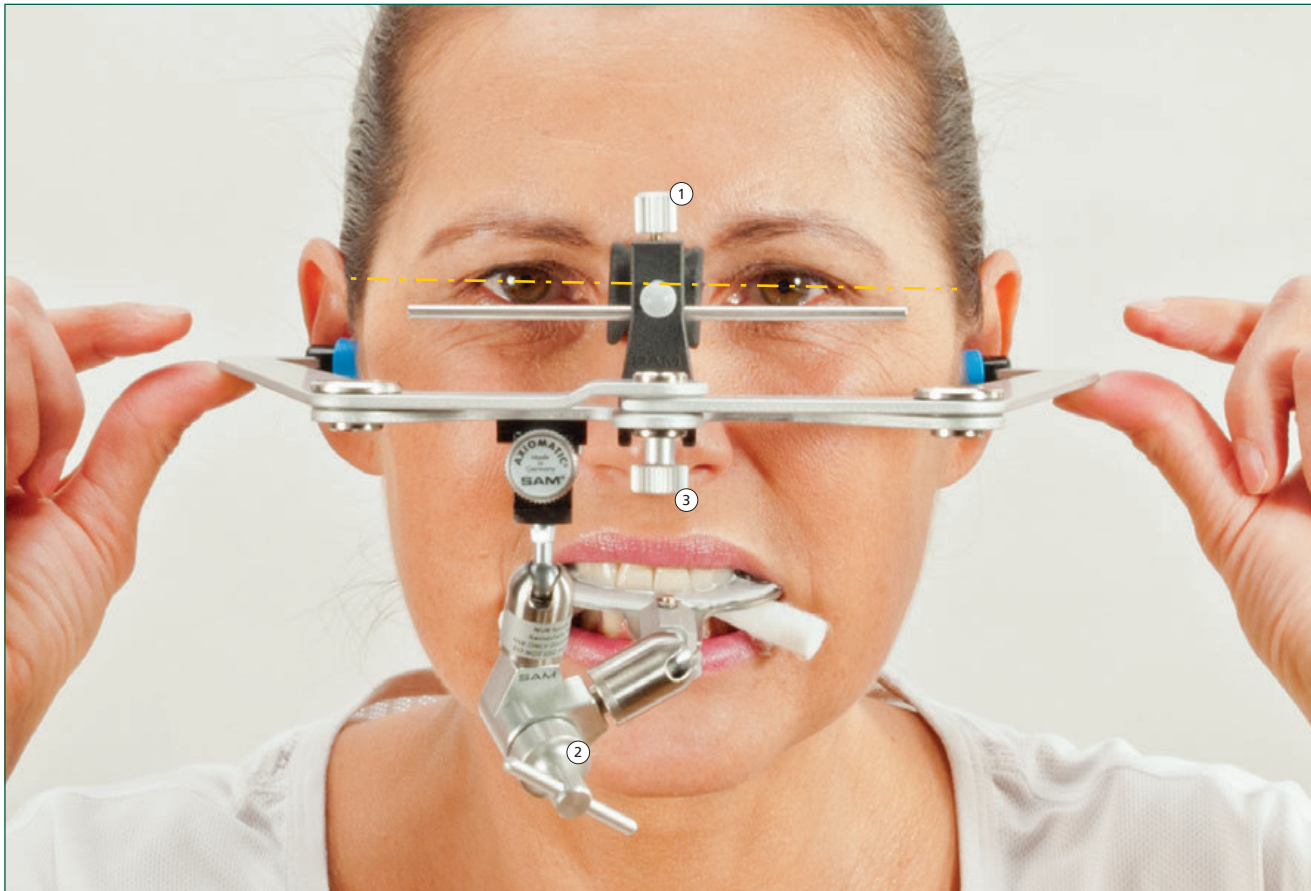
Only when the shoe of the transfer fork assembly is completely plugged, it will tightened with the screw on the transfer fork.

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The interpupillary line leveler assures the correct positioning of the face bow earpieces in the external auditory canals. The patient must always hold the earpieces in position until completion of the face bow transfer and knurled thumb screws on the nasion relator (1) and transfer fork (2) have been set.

It is not necessary to tighten the central thumb screw on the facebow (3) to hold the side arms in position.

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*With the face bow aligned anatomically correct, and secondary check of the interpupillary line leveler, carefully tighten the AX transfer fork assembly cross clamp but **DO NOT OVERTIGHTEN**. Therefore please draw the toggle of the transfer fork assembly only hand tighten. Too large a force application can damage the AX clamp.*

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Connecting the transfer fork assembly NT

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Open the two-toggle transfer fork assembly NT system clamps 1/2 turn, the clamps slide easily from top to bottom and at the correct height to handle the transfer fork.

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The transfer fork adapter is pushed onto the transfer fork AX.

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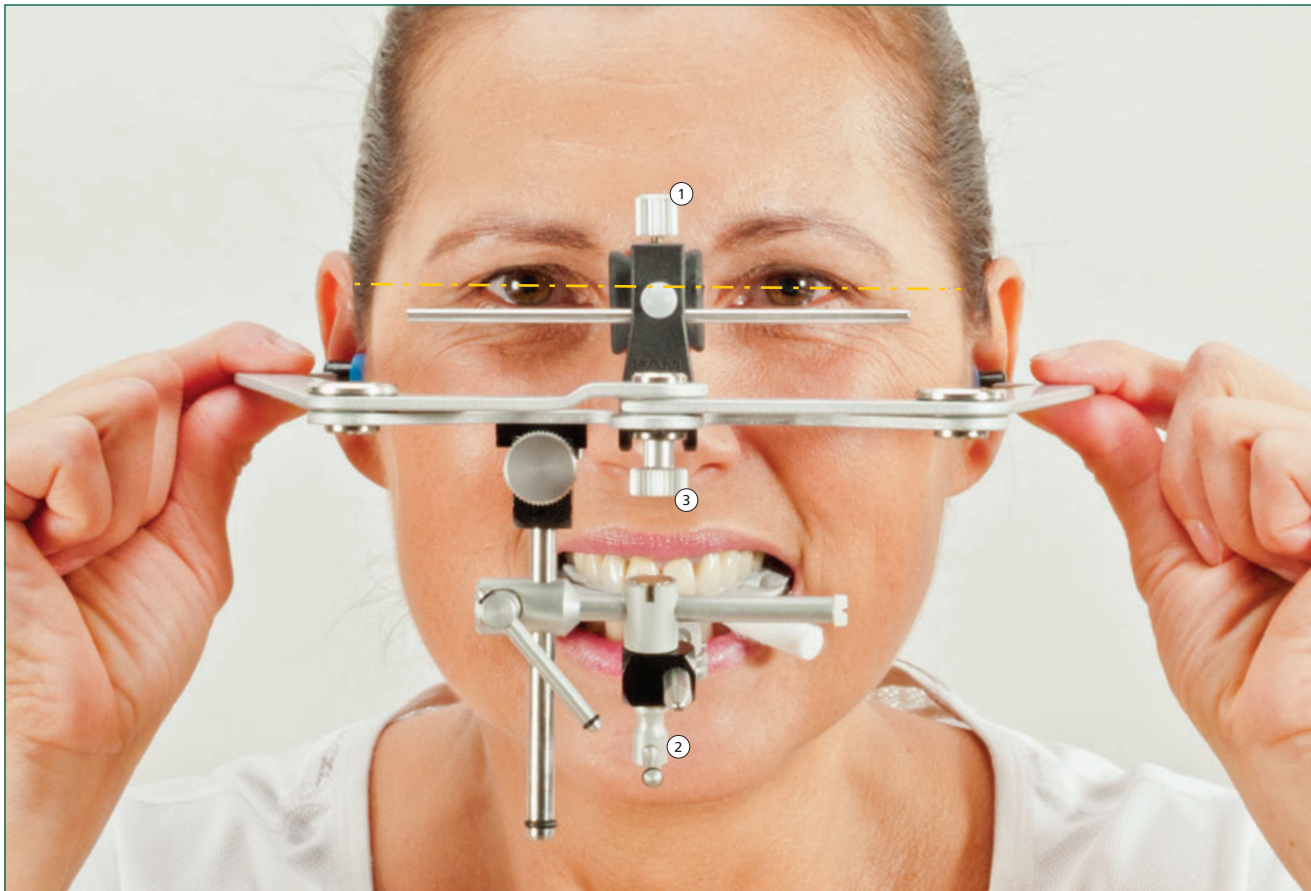
The screw on the transfer fork has to be tighten.

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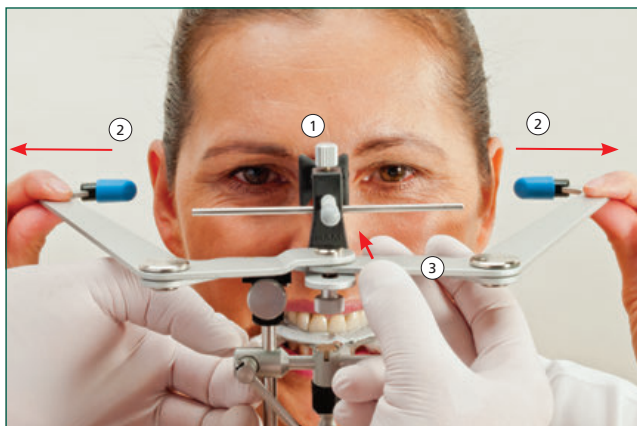
*If the facebow is aligned anatomically correct and the interpupillary line leveler was controlled the second time, the NT-toggle (1) shall be carefully drawn, however, **NOT TOO TIGHT**. Only then the horizontal toggle (2) is tightened.*

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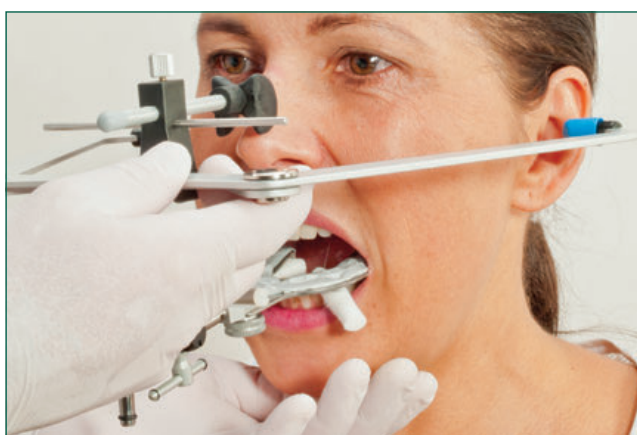
The thumbscrew on the nasion relator (1) is solved. The patient pulls the ear pieces out of the ear canal (2) supported by the frontal compression of the facebow by the examiner.

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
The patient opens his mouth and the examiner guides the facebow down slowly.

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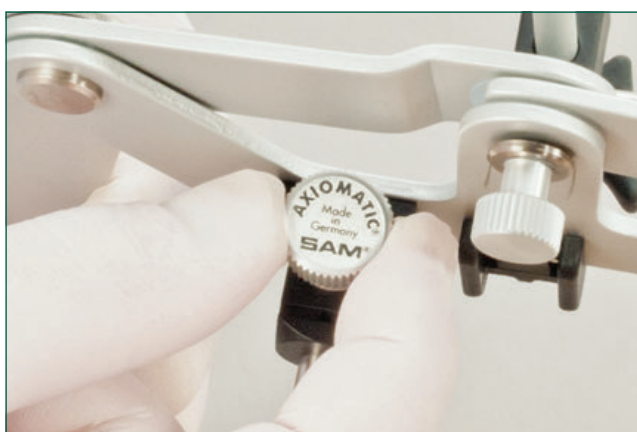The transfer fork is removed together with the bow so carefully, that their position in relation to the facebow does not change.

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For the subsequent indirect upper jaw model mounting with the SAM® transfer stand the transfer fork assembly is removed at the sliding guide.

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If the facebow is positioned properly and the position of the transfer fork is properly secured, the face bow keeps its position without the patient holding it.

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
The AXIOQUICK® face bow transfer can be done quickly and accurately:

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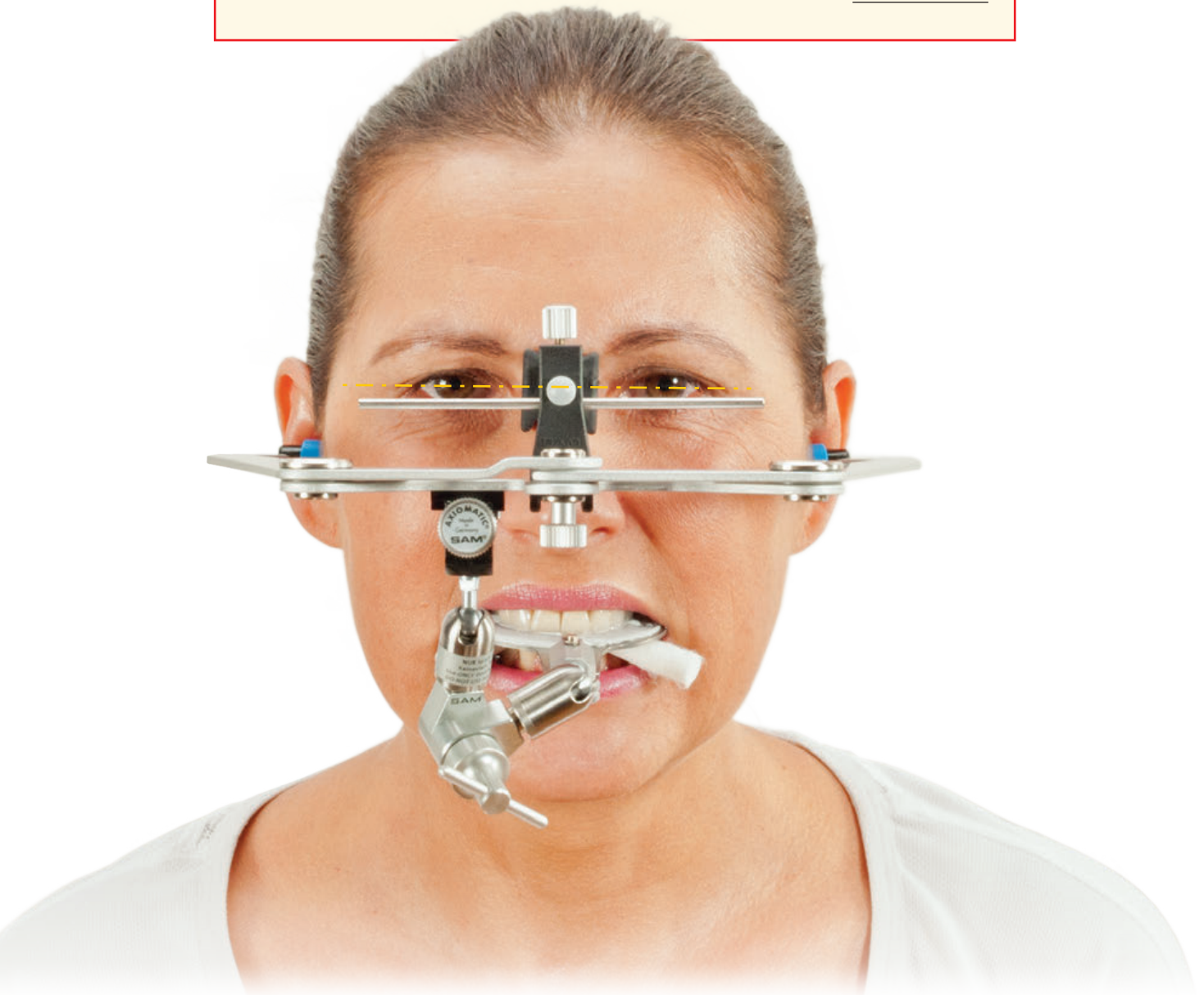Why this alignment technique should be used.

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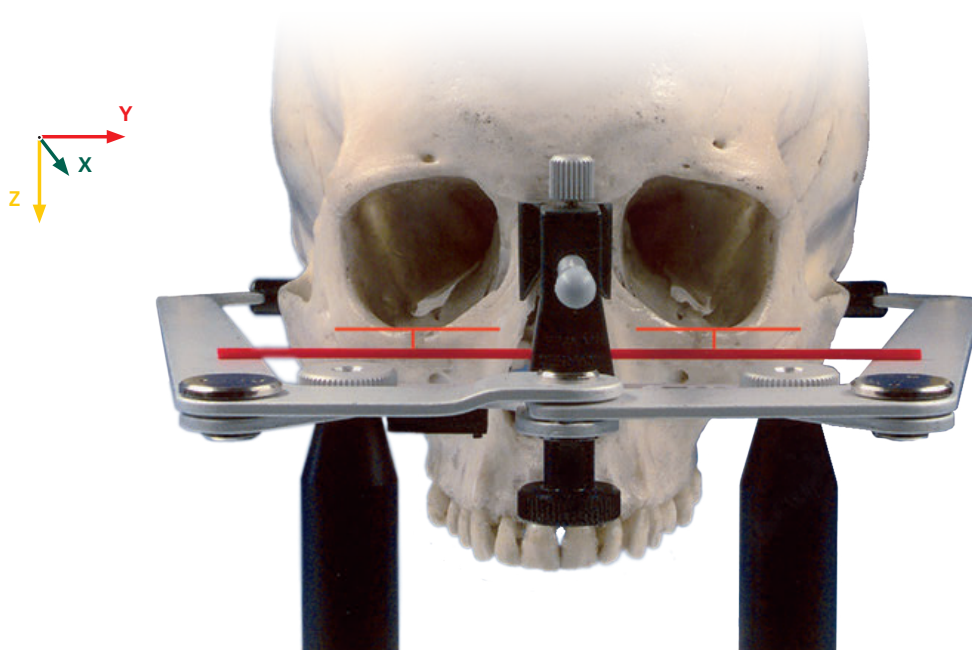
A study was done to determine the mid-sagittal and horizontal plane relationships in reference to the anthropological bony ear canals. The SAM® AXIOQUICK® facebow was modified to attach to a special mounting stand. This facebow was attached to 122 skulls and digitally photographed. It could be noted that the lower limit of the orbital eye sockets were always sufficiently parallel with the horizontal plane of the facebow. A similar result was obtained in another 44 patients, i.e., the horizontal plane was always sufficiently parallel with the Interpupillary plane. The measurement using Photoshop 6.0 was found in 70% of cases side differences of the skull, as well as patients of ≤ 1 mm and lateral deviations of < 2 mm. This study shows that a properly designed facebow with appropriate handling ensures a correct skull related transfer of the maxillary model to the articulator.

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Both in the group of „historic“ skulls, as well as in the group of patients are approximately 70% of the measured side differences ≤ 1 mm. In the larger deviation areas, however, we observed a higher variability in the skulls. In the group of patients were all side differences ≤ 2 mm, in the group of the skull were 8% > 2 mm. The difference in the scattering of the measured values is confirmed by the coefficient of variation.

Bergstrom G.

On the reproduction of dental articulation by means of articulators, a kinematic investigation. Acta Odontol Scan 1950;9 (suppl 4):125-141

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Indirect transfer - Upper Cast Mounting

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The optimal solution for Mounting the upper cast is using the transfer stand AX (ATB 398).

This eliminates the need for using the face bow.

Here the incisal pin is replaced by the transfer stand AX.

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A magnetic block (ATB 338 for MPS and 339 for screw type plates) instead of the mounting plate accommodates for the later set up transfer fork support a bigger and secure support area.

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The transfer fork assembly is inserted in the guide on the upper part of the transfer stand (black) until it stops.

The thumbscrew on the transfer fork assembly is tightened.

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The telescopic transfer fork support (ATB 336) is slid under the transfer fork..

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The articulator is turned over and the thumb screw on the transfer fork support loosened. The discs of the transfer fork slides down until they come into contact with the transfer fork without pressure (if there's not enough space, the rings can be moved laterally such that only small rod comes into contact with the transfer fork). By screwing the thumb screw the transfer fork support is fixed. The articulator can be reversed again.

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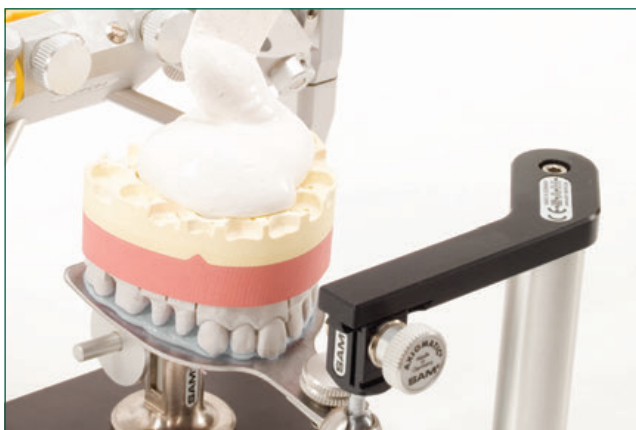
The upper cast is seated into the impression record on the transfer fork. A preliminary check is made to assure enough space for mounting gypsum. Proper installation requires approximately the free space of the little finger (15 to 20 mm). About 15 ml of water to be used for the gypsum per finger thickness.

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
Special low expansion gypsum dental stones are available and should be used. Consistency of lightly whipped cream is suggested. Place a small amount of soft gypsum on the mounting plate surface. Lightly moisten dental cast and place sufficient gypsum on the stone cast to assure contact with mounting plate. Close upper member of articulator to complete gypsum connection. There should be no pressure on closing.

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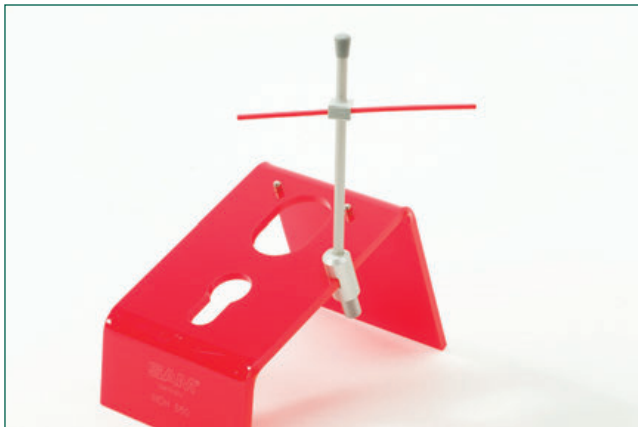Lower Cast Mounting using Mounting Stand

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The mounting stand 560 MOH is used for the mandibular cast mounting and is compatible with all SAM® articulators and laboratory cast mounting instruments.

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Remove and attach incisal pin to upper member of articulator.

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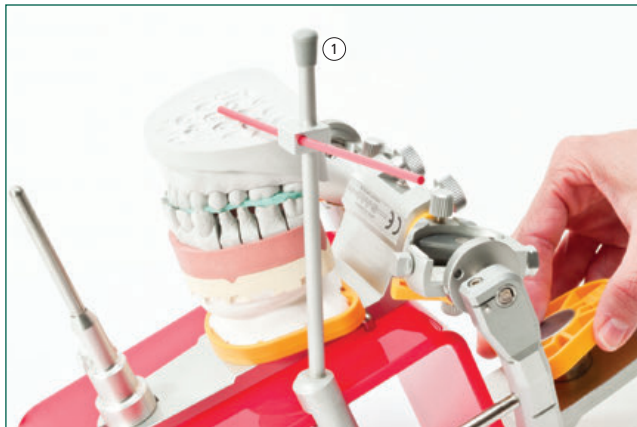
Attach incisal table to lower member of articulator.

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The thickness of the occlusal registration in the premolar region is determined with a caliper and the height of the incisal pin is increased to twice the measured value

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It is strongly recommended to fix centred the models with the intermediate registration with the red flexible plastic rod. Closing rotation is tested whether the model holder (1) does not interfere, and how much gypsum is needed.

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After closing check is done, the mounting plate is filled with a minimum amount of soft gypsum, the lower cast is lightly moistened and covered with sufficient soft gypsum to assure contact with the opposing mounting plate gypsum. The articulator is closed until the incisal pin contacts the incisal table.

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Tip: *If the distance between the mounting plate and the lower cast is too large, we recommend installation in two steps.*

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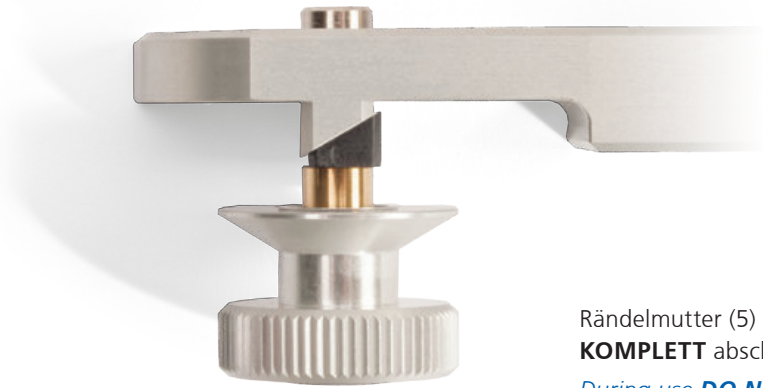
A more detailed manual for model mounting can be purchased on request by SAM or can be downloaded as PDF file.

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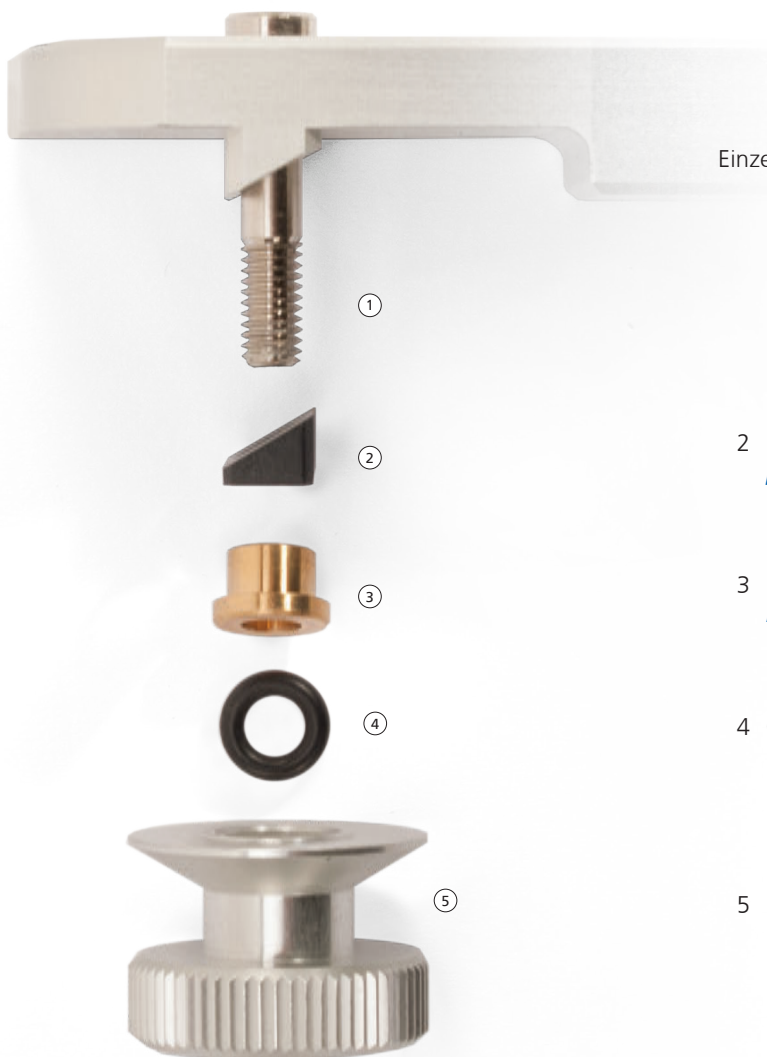
*During use **DO NOT** unscrew thumb nut (5) **COMPLETELY**.*

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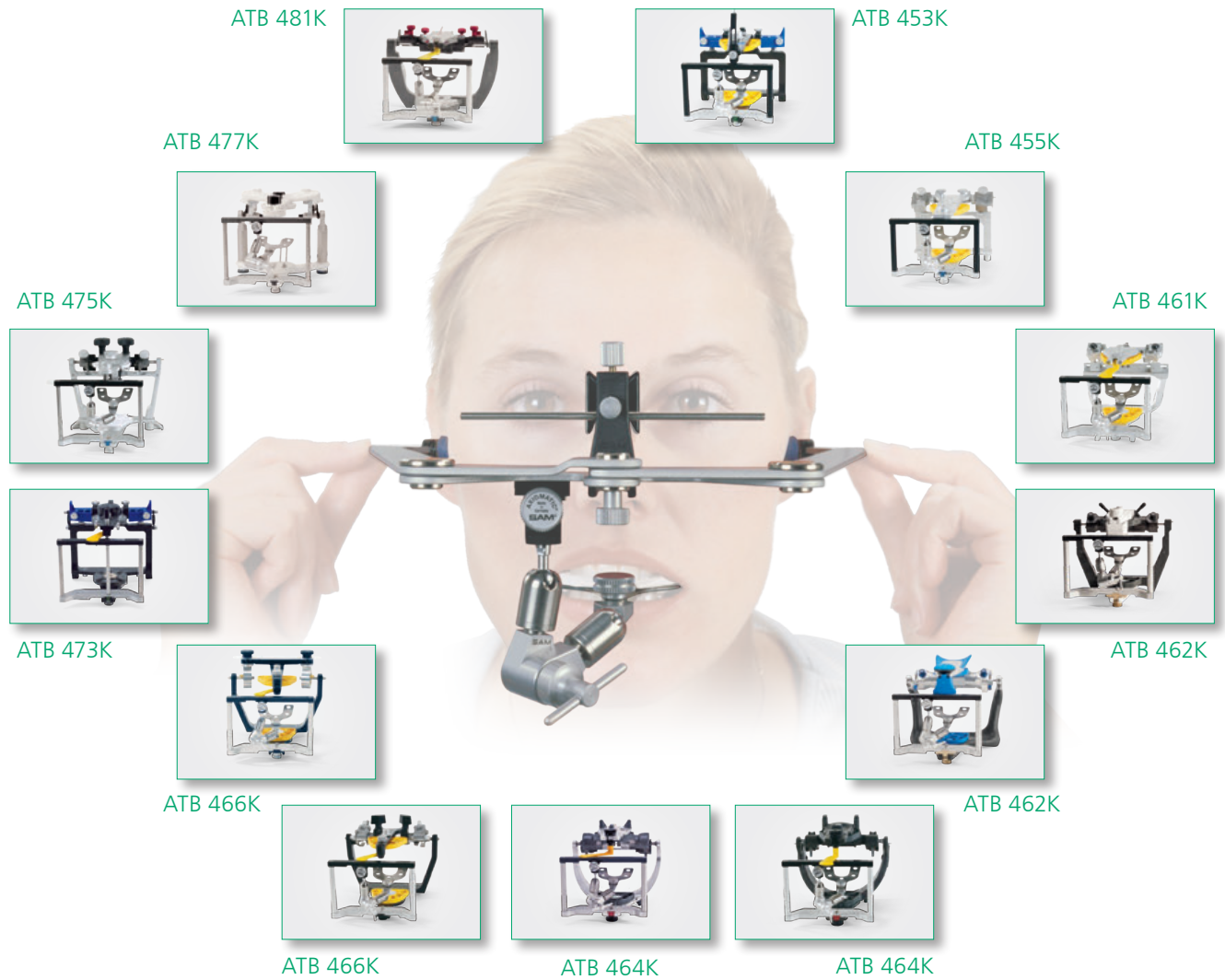
in order sequence to fixed screw of lower part assembly (1).

Rändelmutter (5) **NIE KOMPLETT** abschrauben. *During use **DO NOT** unscrew thumb nut (5) **COMPLETELY**.*

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
Use for AXIOQUICK® in special transfer stands for non -SAM® type articulators

<p>ATB 453K Transferstand - P für Panadent (niedrig) <i>transfer stand - P for Panadent (low)</i></p>	<p>ATB 455K Transferstand - WM 8 für WhipMix 8000 Serie <i>transfer stand - WM 8 for WhipMix 8000 series</i></p>	<p>ATB 461K Transferstand - D für Denar <i>transfer stand - D for Denar</i></p>	<p>ATB 462K Transferstand - G für Girrbach Artex und Artex Carbon <i>transfer stand - G for Girrbach Artex and Artex Carbon</i></p>	<p>ATB 464K Transferstand - K für KaVo Protar und EVO <i>transfer stand - K for KaVo Protar and EVO</i></p>
<p>ATB 466K Transferstand - H für Hanau Modular, Wide-Vue <i>transfer stand - H for Hanau Modular, Wide-Vue</i></p>	<p>ATB 473K Transferstand - WM2PH für WhipMix 2000 Serie und Panadent <i>transfer stand - WM2PH for WhipMix 2000 series and Panadent</i></p>	<p>ATB 475K Transferstand - WM 3 für WhipMix 3000 Serie <i>transfer stand WM 3 for WhipMix 3000 series</i></p>	<p>ATB 477K Transferstand - ST1 für für Stratos 100 <i>transfer stand ST1 for Stratos 100</i></p>	<p>ATB 481K Transferstand - D3 für Denar 300 Serie <i>transfer stand D3 for Denar 300 series</i></p>

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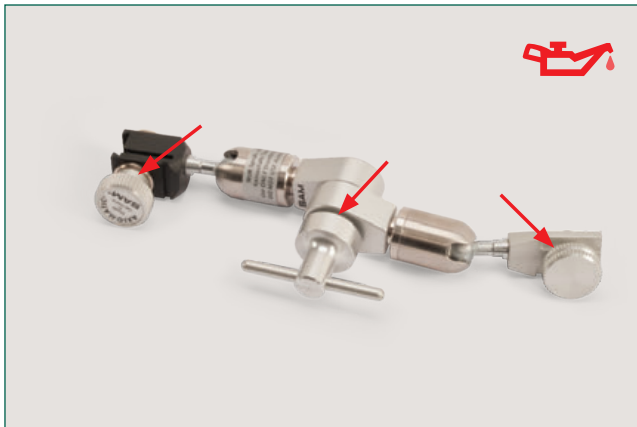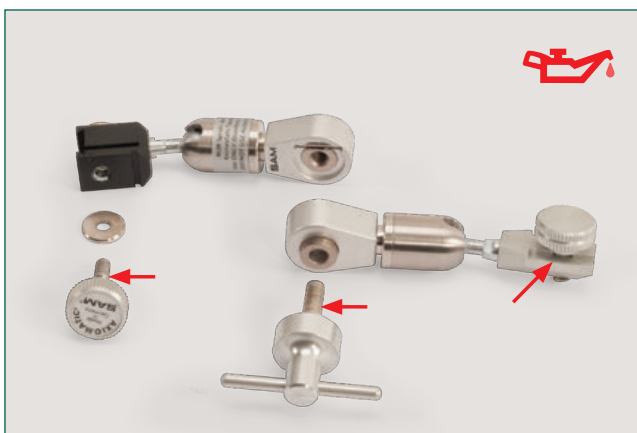
The internal and external threads of the transfer fork assembly should be periodically lubricated with dental hand piece oils or similar type of lubricants

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*Open the toggle and fixing screw of the transfer fork assembly AX.
 Remove carefully any impurities.
 Place drop of oil **only** on the thread screws of the transfer fork assembly.*

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Note reassembly to the fact that the protruding edge of the joint has a precise place in the indentation and fitting should require no force.

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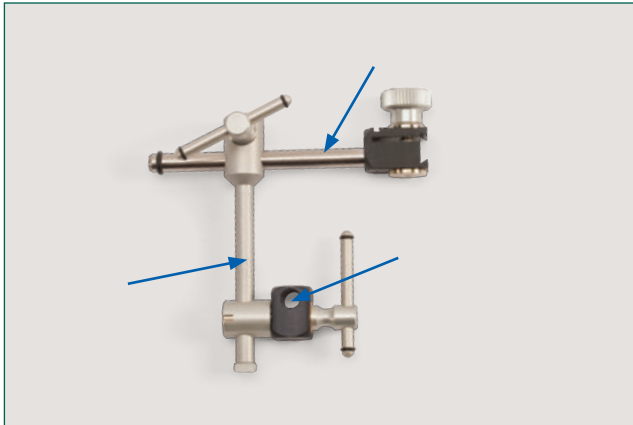
Note:

Never tighten thumbscrews more firmly than needed.
 Sometimes less is more.

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
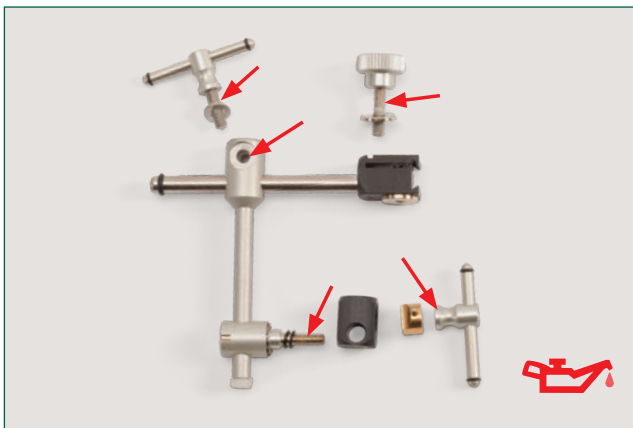
The blue marked rods and holes for the transfer fork must be absolutely grease-free. They should be degreased periodically with alcohol to preserve the proper friction effect for clamping.

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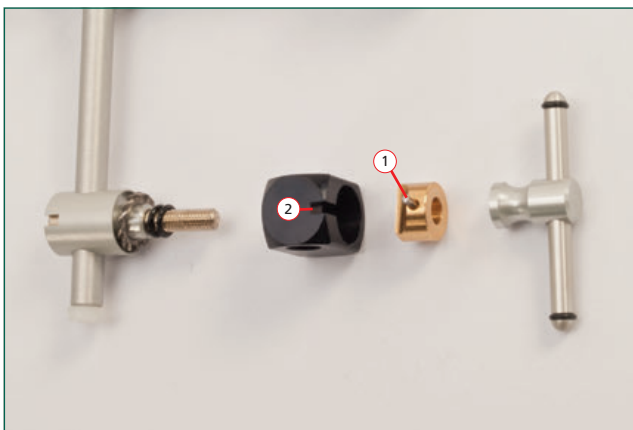
The internal and external threads of the transfer fork assembly should periodically be lubricated with dental hand piece oils or similar types of oils.

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The diagram on the left shows the proper assembly of the parts in the clamping mechanism. Notice that the cylinder pin completely should fit in the notch. Only then the transfer fork assembly may be screwed together again.

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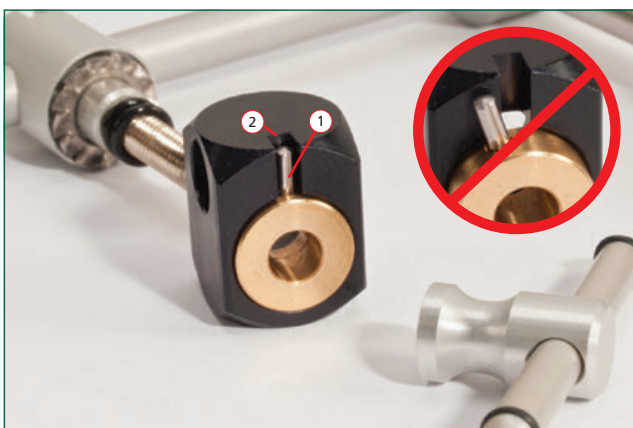Note:

The black clamping mechanism does not work unless there is a transfer fork handle in the slot.

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Cleaning, disinfection and sterilization

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Prior to cleaning please remove the transfer fork from the transfer fork assembly and remove the hygienic ear piece caps from the face bow earpieces.

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Disposable

*The blue hygiene protection caps are designed **for the single use** only and should be disposed after use like the disposable gloves.*

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Only the transfer forks ATB 396 and AXO 230 should be sterilized in the Autoclave

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The transfer fork is classified as a „semicritical medicine product“ (according to MPG) and must be sterilized as such. For this the steam sterilization is recommended in the fractionated vacuum procedure for 134 degrees centigrade for 10 minutes in a device after German Institute for Standardization DIN EN ISO 17665-1.

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It is highly recommended to use some type of cloth wipes for cleaning the SAM® face bow and transfer fork assemblies.

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Facebow, transfer fork assembly, adaptors and nasion relator should NEVER be placed in chemical solutions or ultrasound.

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hygienic earpiece caps

(200 pieces) disposable blue caps that go on the black plastic earpieces on the AXIOQUICK® earbow and the alignment flags of the AXIOGRAPH® III, used to cover the black plastic earpieces on the AXIOQUICK®, and AXIOGRAPH® III. The Earpiece caps assure proper and comfortable positioning in the ear.

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transfer fork NT

two-piece, composed of transfer fork AX (ATB 396), used in transfer fork assembly AX and transfer fork adapter (ATB 326), for use in AXIOQUICK® I-NT (ATB 305) and AXIOGRAPH®

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transfer fork adapter NT

connects transfer fork AX (ATB 396) to transfer fork NT

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NT - clamp upgrade kit

used for conversion of SAM® anatomic transfer facebows and transfer fork assemblies equipped with regular double clamp to non-wrench NT type clamp

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transfer fork AX

for AXIOMATIC® transfer fork assembly, removable for transportation and to reattach securely, sterilizable

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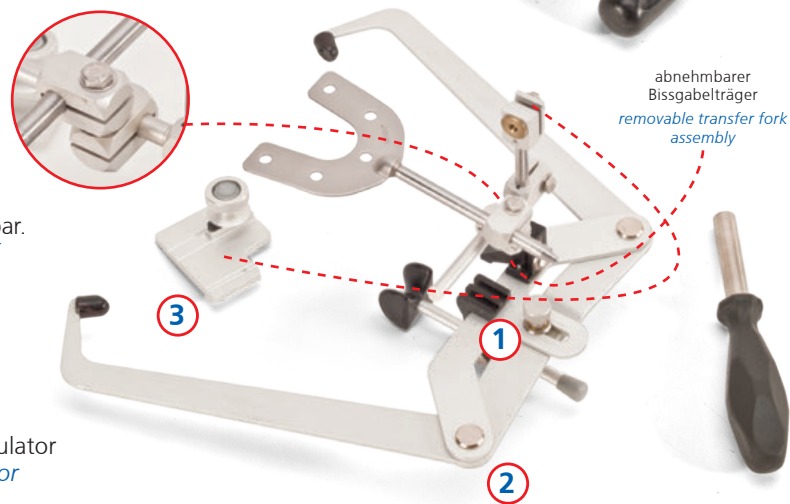
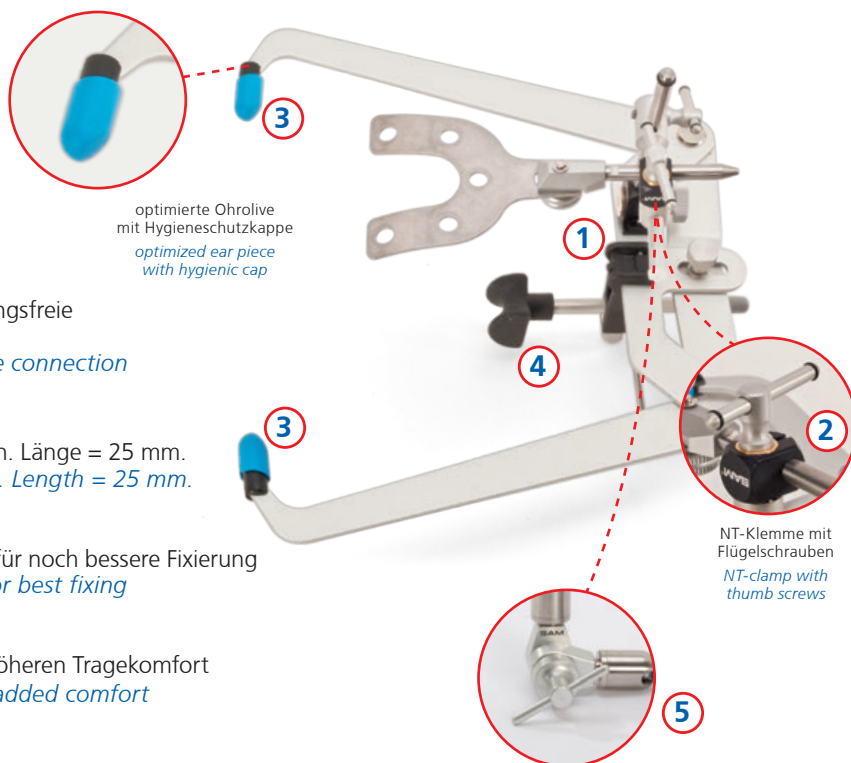
Experts provide up to date first hand information on the SAM® System to Dentists, Assistants, and Technicians. Theoretical didactic and practical exercises are the core of our training concept.

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In our training center in Gauting, near Munich, we would like to give you in a small course on the understanding of our system and its use in dentistry. Naturally we consider the different levels of knowledge of the participants in the course schedule. At the end any individual questions will be answered.

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SAM® Präzisionstechnik GmbH