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is a transparent occlusal plane indicator according to Gutowski Stegmaier, it is designed for the quick and easy production of occlusal splints. The splints are made from either auto-polymerizing or light-cured composite.

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*Work is completed without inter-
ruption (such as curing).*

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*No distortion or bite raising from
using polymerized methyl metha-
crylate acrylics (PMMA, hot or cold
curing acrylics).*

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Securely held with single toggle clamp

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***AVAILABLE AT SELECTED AND
AUTHORIZED DEALERS***

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The SAM Axiosplit GS was developed as the result of decades of practical experience by the well known dentist - dental technician duo Gutowski / Stegmaier. In many cases the demand of an appropriate aid for splint manufacture has been recognized and implemented in collaboration with the SAM® Präzisionstechnik GmbH in 2014.

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Born 1936 • Promotion 1960 • since 1962 practice in Schwabisch Gmund • Between 1966 and 1976 Dr. Lauritzen, Dr. Thomas, Dr. Stuart, Professor Goldman, Professor Tanner • 1992 Professor at the University of Zagreb / Croatia • Book author and author of significant scholarly publications • Member and honorary member of numerous scientific societies • More than 900 training courses with over 40,000 participants

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Born 1966 • 1983 (teaching as a dental technician in the dental laboratory Schenk in Waldstetten) • Training Prof. Gutowski, Ch. Berg, E.Steger, Hegenbart, Polz, A. Schenk, Prof. Edelfhof, Dr. Borchard • 1995 Master Dental Technician in Master School Munich • 1995-1999 Head of the Education Department in the dental laboratory Schenk in Waldstetten • 2000 Practice Laboratory Gutowski • training technician and contributor for splints, crowns, partial crowns, full ceramic and combined courses • Development for SAM: cone pins and Axiosplint GS

VOCATIONAL TRAINING OF THE HIGHEST LEVEL

**Also Courses about
splint making
with J.Stegmaier**

The CDE has established itself as a forum for competent and academically advanced dental training/ further education. The participant benefit from the highest quality of equipment, comfortable ambience and excellent supervision. Many renowned companies and institutions in the dental field make use of our institute for vocational training and other events.

*Informations about our
current course offers:*

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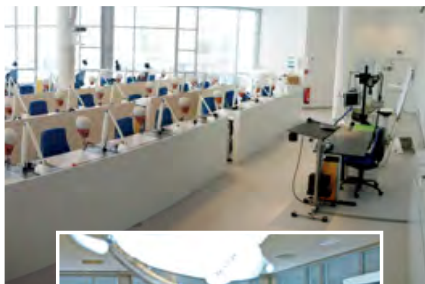
STEP BY STEP INSTRUCTIONS HOW TO MAKE A TEMPORARY SPLINT USING THE AXIOSPLINT GS

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The following description describes how to make a time-saving splint primarily for bruxers.

The dentist had previously taken an impression and raised the bite. About 2 hours for a dental technician anatomically correct tooth morphology on a temporary splint.

To use the AXIOSPLINT GS you need a SAM articulator equipped with AXIOPLIT® system.

Patient, männlich, ca. 37 Jahre, weist massive Schäden...
Patient, male, approx. 37 years, has severe tooth abrasion...

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Before splint making, the plaster model is mounted on the articulator. You can do this step by step following our "Model Mounting" instructions in our articulator manual.

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Cover the wax-up with transparent silicone (for example, R-SI-LINE® EXTRA BITE of R-Dental).

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*Remove the dental cast with wax up and silicone cover.
Place in pressure pot to complete the impression molding.*

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The wax occlusal surface is removed from the model with a steam jet or hot water in the pressure pot.

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The model is checked and cleaned. Alginate separating liquid is painted on the upper cast, prior to remounting on the articulator.

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Light-curing resin-based composite (dentine) is applied to the dental stone cast with the spatula.

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The composite resin (enamel) is also placed into the transparent silicone impression.

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*Replace articulator upper member into position on the lower
member and lock into the Centric position.*

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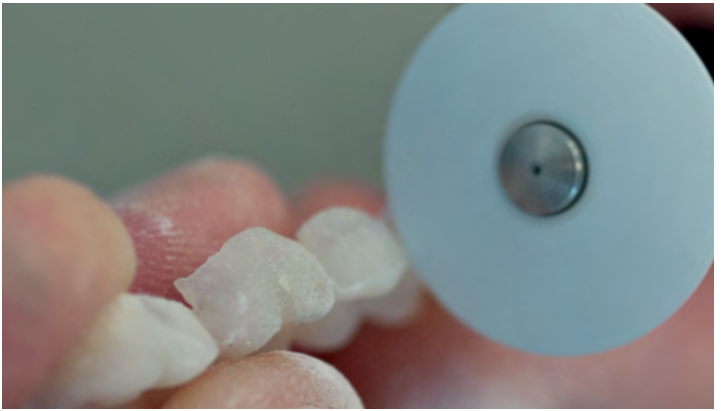
The gypsum dental cast and Axiosplit GS are assembled...

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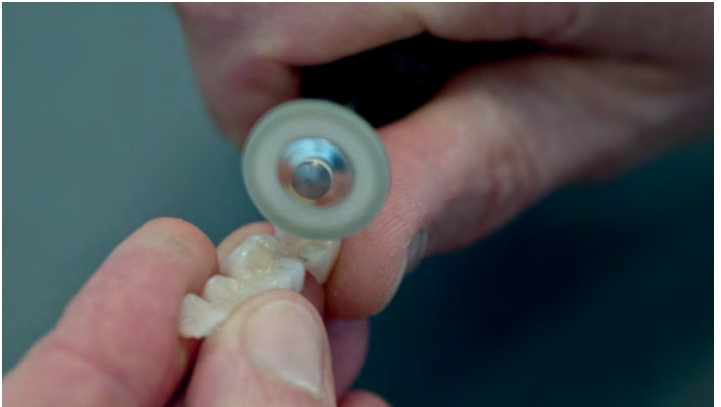
The temporary splint (three parts) is finished with a white rotating silicone wheel.

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The splint is polished with polishing paste and a rotating goat hair brush.

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With carbide and diamond burs the perfect fitting occlusion is ground on the temporary splint.

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Tip: Set the incisal pin very precisely with the micrometer screw (ART 533) while grinding

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To check and adjust the protrusion pathway use the protrusion screws.

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The semi-permanent (three parts) splint is now fabricated and ready for fitting.

Fertig hergestellte semipermanente Schiene in 3 Teilen
Ready made temporary splints in 3 parts with contact points.

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A special compilation for the courses of Gutowski and for daily working

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*See an example use of the
AXIOSPLINT as mp4-movie.*

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Fußbergstraße 1 82131 Gauting Germany
Tel: +49 (89) 800 654 - 0 Fax: +49 (89) 800 654 - 32
Email: info@sam-dental.de

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are registered trade marks of the SAM® Präzisionstechnik GmbH
R-dental Dentalerzeugnisse GmbH
sind eingetragene Warenzeichen der SAM® Präzisionstechnik GmbH