HANDBUCH / *MANUAL*

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MODEL MOUNTING
with the laboratory cast mounting instrument

SAM® • AXIOGRAPH® • AXIOTRON® • AXIOQUICK®
AXIOQUICK® RECORDER • AXIOSAW® • AXIOPIN® • AXIODRILL®
AXIOSIM® • AXIOCOMP® • AXIOWAX® • AXIOSPLIT®

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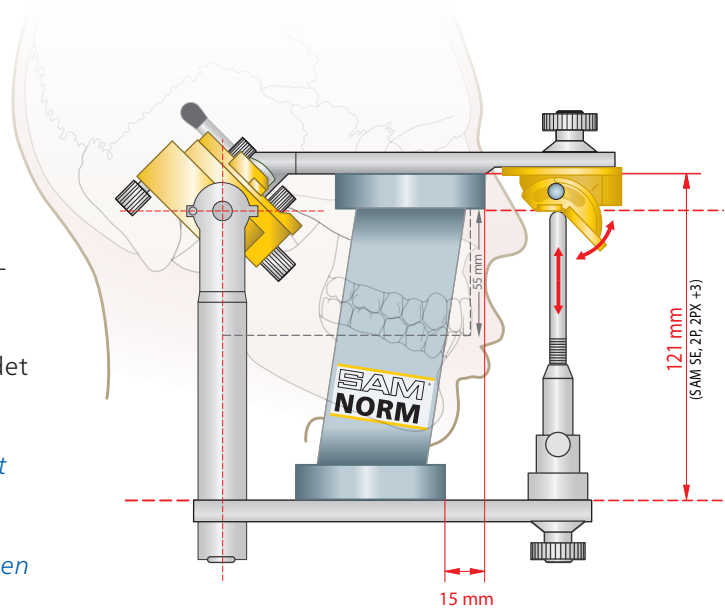
All SAM devices and articulators with the P-height (since the end of the 80s standard) have the same interior height of 121 mm - the „SAM Standard“. Therefore models can easily be exchanged between the individual devices.

Also many auxiliary devices and additional components can be used in all devices equally.

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*The compatibility between the individual devices, the many special instruments and the various special solutions for all areas of dentistry and dental technology and the knowledge of the practice round off what has been known as **SAM SYSTEM** for almost half a century*

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The model mounting described in the following manual has been carried out as an example with the laboratory cast mounting instrument, but it can easily be done in the same way with all other SAM articulators.

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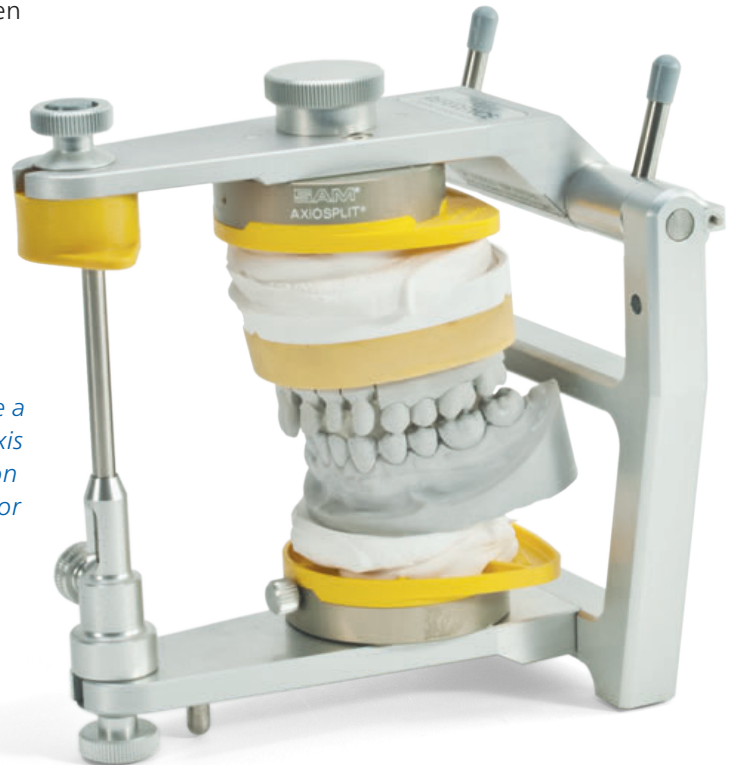
During mounting the centric lock of the articulators should be closed. For more detailed information on the programming and possible settings of the articulator, please refer to the respective manuals, which are enclosed with the articulators or ask directly at SAM.

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The cast mounting instrument was designed to assure a high-precision opening and closing around a rotation axis without errors during the cast mounting. This precision instrument eliminates the need to use an articulator for cast mounting, avoids very common articulator condylar housing damage caused by plaster, and allows articulator to be free for other work.

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All mounting plates are designed for single use.

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A conversion from one cast mounting system to the other via different additional kits is easy to handle for users.

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MPS 150 MPS Montageplatten, gelb (20 Stück) inklusive Haftplatten
MPS mounting plates, yellow (20 pieces) including metal adhesion plates

MPS 155 MPS Montageplatten, gelb BP (100 Stück) Großpackung, inklusive Haftplatten
MPS mounting plates, yellow BP (100 pieces) bulk pack, including metal adhesion plates

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The maxilla and mandible are related to the cranium and also to each other. This three-dimensional position should be transferred to the mounting instrument or to the articulator to simulate the patient. This can be done very accurately with the AXIOQUICK® Anatomic Esthetic related transfer bow. This SAM® esthetic anatomic facebow orients itself on its positioning to three bony reference points (Porion, Nasion, Porion) and thereby forms a reproducible reference plane.

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More detailed information on facebow transfer can be found in the corresponding manual.

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Prior to cast upper cast mounting, remove the incisal pin.

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Attach AX (ATB 398) to the lower member of the instrument.

The incisal table remains attached to the upper member..

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A magnetic block ATB 338 (or ATB 339 for screw mounting plates) can also be used when using the screw mounting system or the SAMS magnetic mounting system (as shown in the figure).

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This block is placed in the lower part of the articulator instead of the mounting plate.

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On this base the telescopic transfer fork support is placed. Depending on the position of the fork, the discs are used or not (see 1.09 and following).

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With the block there is a larger base area in the lower part of the articulator, on this the transfer fork support can be positioned much more flexibly.

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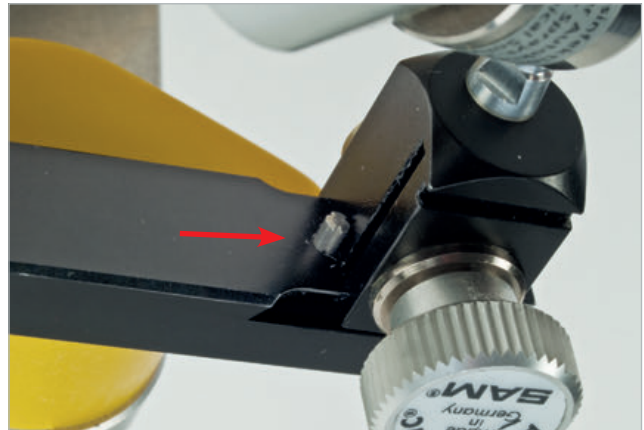
Push the transfer fork assembly and the transfer fork on the dovetail attachment of the transfer stand and tighten the fixing screw.

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Attention: The transfer fork assembly has to be pushed in until it stops and fixed.

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
The telescopic transfer fork support ATB 336 is used to hold and stabilize the transfer fork assembly firmly in position against the pressure of the model and plaster during the attachment of the upper cast.

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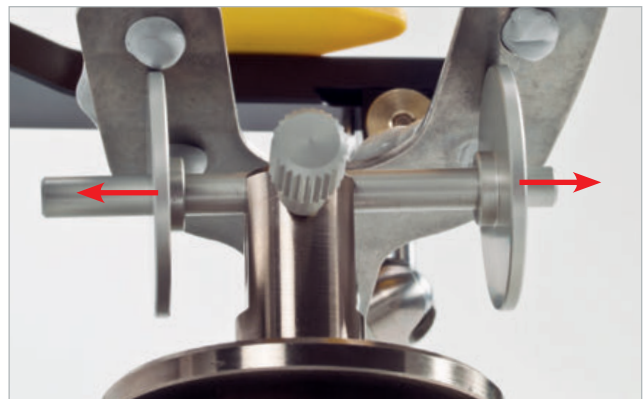
The disks of the transfer fork support supports a higher standing transfer fork. The discs are adapted to the transfer fork by means of a horizontal movement. If the upper jaw is placed very far down in the skull and thus the model also far down in the cast mounting instrument, the discs

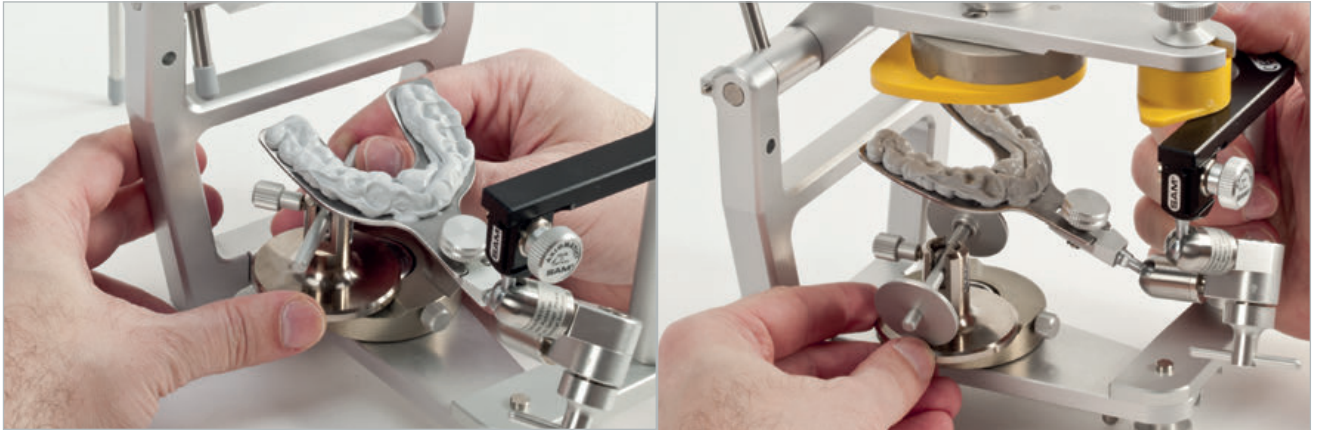
may have to be removed; These can easily be pushed on and off again. The transfer fork should be supported as centrally as possible.

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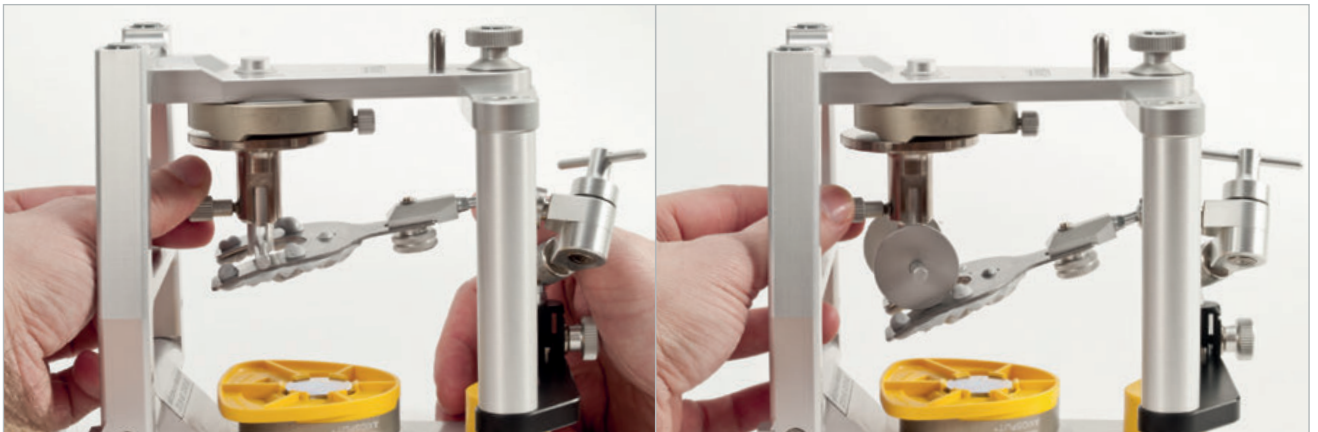*The lowered transfer fork support is carefully pushed under the transfer fork.
The transfer fork is to be supported in the middle as shown here with thumb screw positioned to the back.*

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
Invert the instrument and with the screw loosened, the telescopic support can be easily (without pressure) positioned onto the under surface of the fork. Once seated, without pressure, the thumb screw is tightened you dental cast is firmly held in position.

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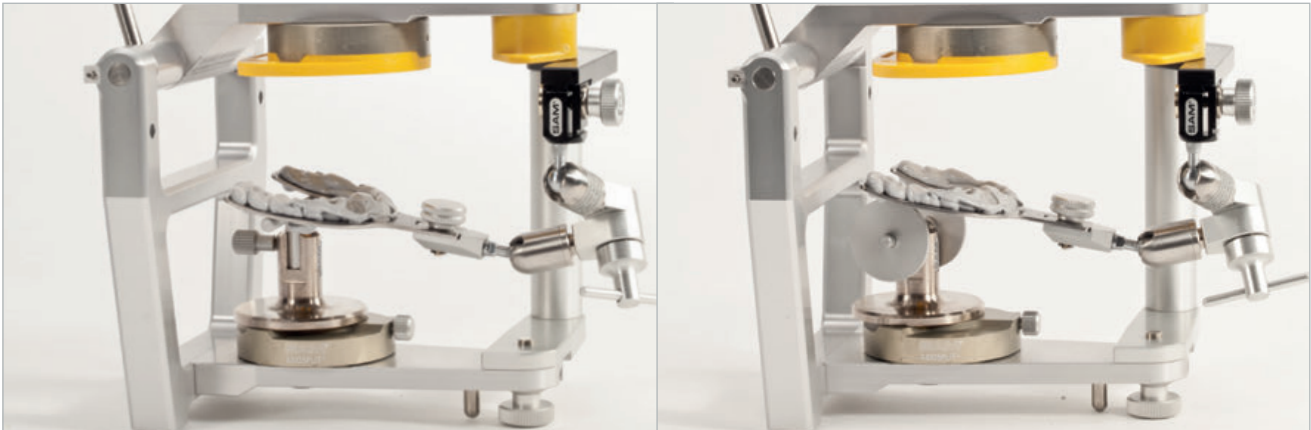The support is tightened with the screw not too firmly, as otherwise the transfer fork support could twist.

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After reversing the instrument, the ATB 336 supports the transfer fork without changing its position. The transfer fork should be supported in the middle.

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If the articulator is flipped open, it is supported with the vertical support rod (ART 525). Therefore it is placed on the thumbscrew of the incisal table.

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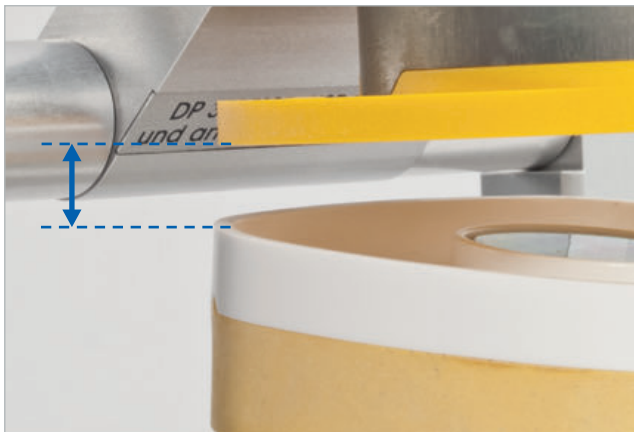
Flip open the upper member as shown here.

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The upper model is positioned in the impressions of the transfer fork. The upper cast should seat without movement. Otherwise model and coating must be checked for errors.

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Examine space between upper cast and mounting plate to determine correct amount of mounting plaster to be used.

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Mix the mounting plaster creamy according to manufacturer's instructions.

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Always place a small amount of mounting plaster onto the upper mount plate while it is still moist and also flows very easily to assure a good attachment to the plate.

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The upper jaw model is the same procedure. Models with plaster socket should be moistened slightly on the surface, so that the fresh plaster is better connected to the model.

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If the freshly applied plaster loses its luster, the device is closed. When the surplus plaster is spread, the model could be moved.

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The device is opened and the transfer stand and the transfer fork assembly is removed.

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When mounting the lower jaw model, a centric registration may be used to accurately position the lower jaw in relation to the upper jaw.

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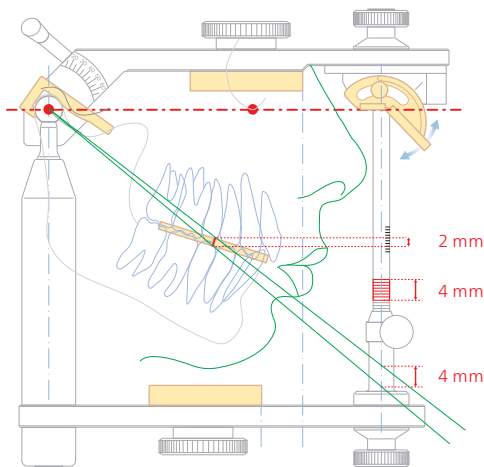
Measure the centric record using a caliper. The thickest part is usually located in the premolar region.

In this example, the value 2 mm is measured.

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The height of the incisal pin is set to twice the value of the registered thickness of the record in the premolar region. This will compensate the difference in height after withdraw the registration.

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$2 \times 2 \text{ mm} = + 4 \text{ mm}$

eingestellt.

In the example case, the incisal pin is

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Following the lower jaw model mounting the registration is removed. The pin should then indicate zero.

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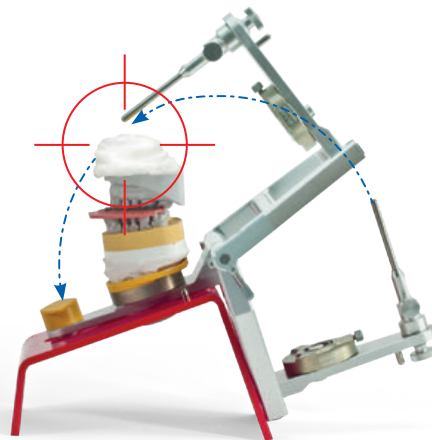
To facilitate the lower jaw model mounting we recommend the red mounting stand MOH 560.

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
Caution - in worst case the incisal pin can collide with the plaster while closing the device.

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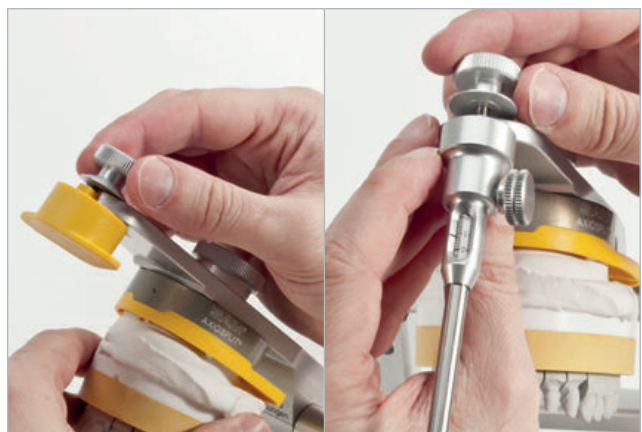As a precaution the incisal table in the upper part is replaced with the incisal pin in the lower part for lower jaw mounting. After the mounting, table and pin can be quickly re-exchanged.

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
The mounting plate is inserted into the lower part

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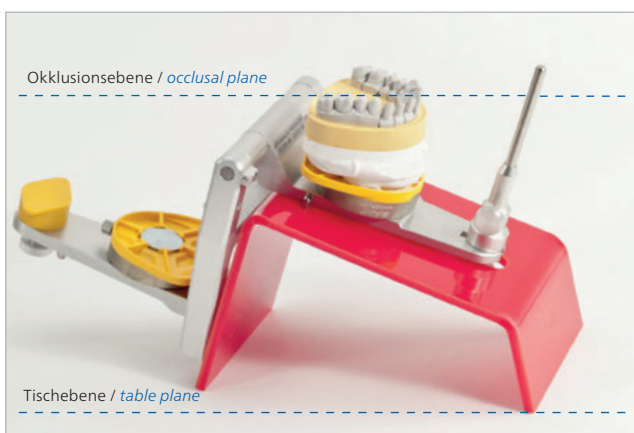The cast mounting instrument is attached into the mounting stand, the upper part thumb screws slip in between the slots.

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The mounted upper jaw model is inclined downwards due its jaw position (Frankfurt Horizontal). The cast mounting instrument compensates for this, so that the upper cast is at least horizontale to the table plane. Thereby, the lower cast is stable and there is no flow from the plaster model.

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The cast holder is placed on the right side and is fixed with the knurled screw on the lower end.

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The red elastic plastic rod is pushed through the slide of the model holder. The rod tip is rotated centered on the model. Lowering the metal slide containing the red plastic rod, allows for proper positioning of the rod in the central area of the lower cast with sufficient pressure to hold the cast in position.

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The mounting instrument is closed as a trial. The amount of required plaster is estimated by the distance between the yellow mounting plate and dental cast.

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*Mix mounting plaster creamy according to manufacturer's instructions.
The plaster is applied cloudy to the mounting plate.*

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The procedure is the same with plaster models. These should be slightly moistened on the surface, so that the fresh plaster is better connected with the model.

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The mounting instrument is closed and the plaster is allowed to set.

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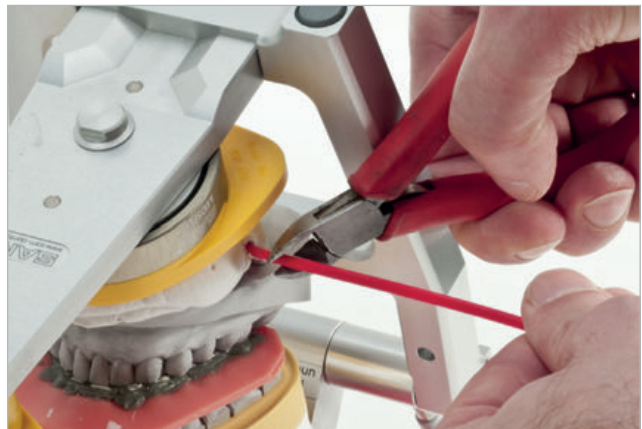
After the plaster has set the cast holder is removed from the instrument.

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The red elastic rod can be cut off with a pair of side cutters or a dental burr.

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The instrument is removed from the mounting stand.

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After removal of the centric record the incisal pin is reset as shown..

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The incisal pin slides back to its original position (at Zero).

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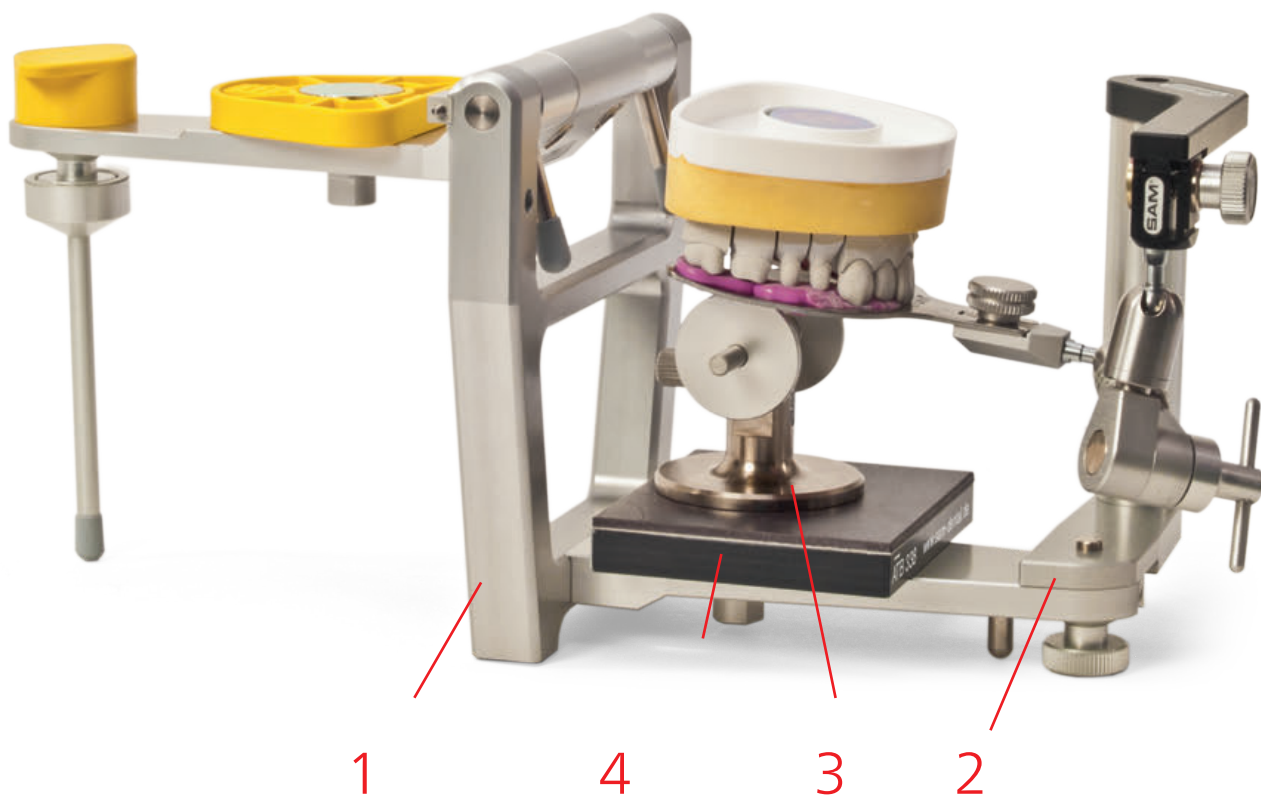
The mounted can be attached to any SAM® Articulator.

DIE PERFEKTE MODELLMONTAGE

Oberkiefermontage - Upper Cast Model Mounting

1 Einartikulier- Instrument II
MOH 700 / MOH 715M / ASP 350
*Laboratory cast mounting instrument II
MOH 700 / MOH 715M / ASP 350*

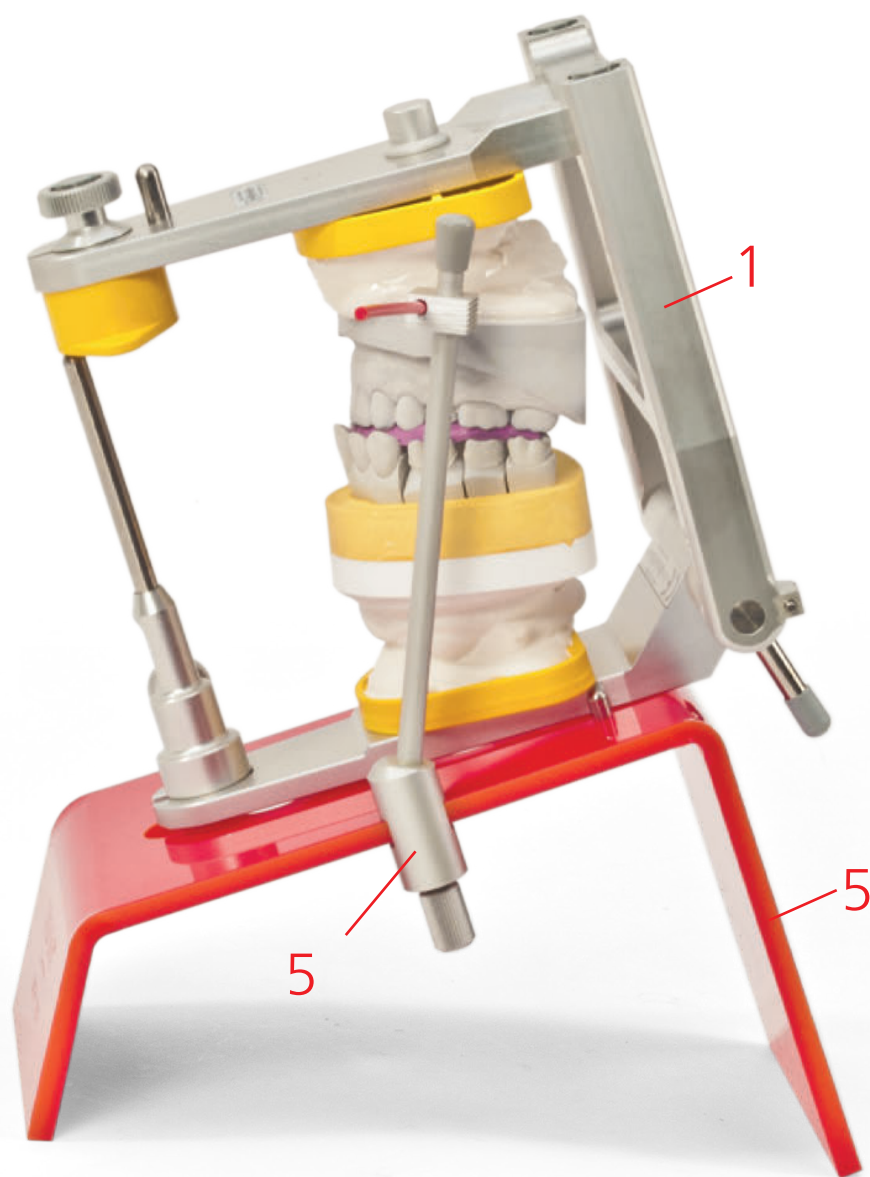
2 Transferstand AX ATB 398
Transferstand AX ATB 398

3 Teleskop- Bissgabelstütze ATB 336
Transfer fork support, telescopic ATB 336

4 Magnetsockel für Bissgabelstütze
ATB 338 für MPS-System
ATB 339 für Schraubmontageplatten-Systeme
*magnetic block for transfer fork support
ATB 338 for MPS-system
ATB 339 for systems with screw type mounting plates*

THE PERFECT MODEL MOUNTING

Unterkiefermontage - Lower Cast Model Mounting

- 1 Einartikulier- Instrument II
MOH 700 / MOH 715M / ASP 350
Laboratory cast mounting instrument II
MOH 700 / MOH 715M / ASP 350

- 5 Montagestand, inklusive Modellhalter
und 30 Elastikstäbe
MOH 560
mounting stand includes cast holder and 30
flexible plastic rods
MOH 560

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vertical support rod

Extension rod for supporting the unfolded articulator upper part

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incisal pin IIIPM, micro adjustable, complete

Precision incisal pin complete with knurled screw for all SAM® articulators with micro scale for reading into 0,1 mm steps for all SAM® Articulators

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magnetic block for transfer fork support

mounting plate for stabilization of the transfer fork support (ATB 336) in an articulator

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transfer fork support, telescopic

used to support facebow transfer fork during upper cast mounting procedures

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transfer stand AX

for an easy and secure upper cast model mounting in all SAM® articulators (P-Norm) and in all reference centric instruments

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transfer fork assembly AX - AXIOMATIC

with a single secure universal clamping mechanism and removable transfer fork AX

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mounting stand

Auxiliary device for an easier lower jaw model assembly, including model holder and 30 flexible plastic rods

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
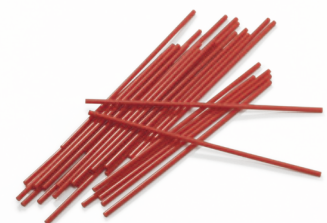
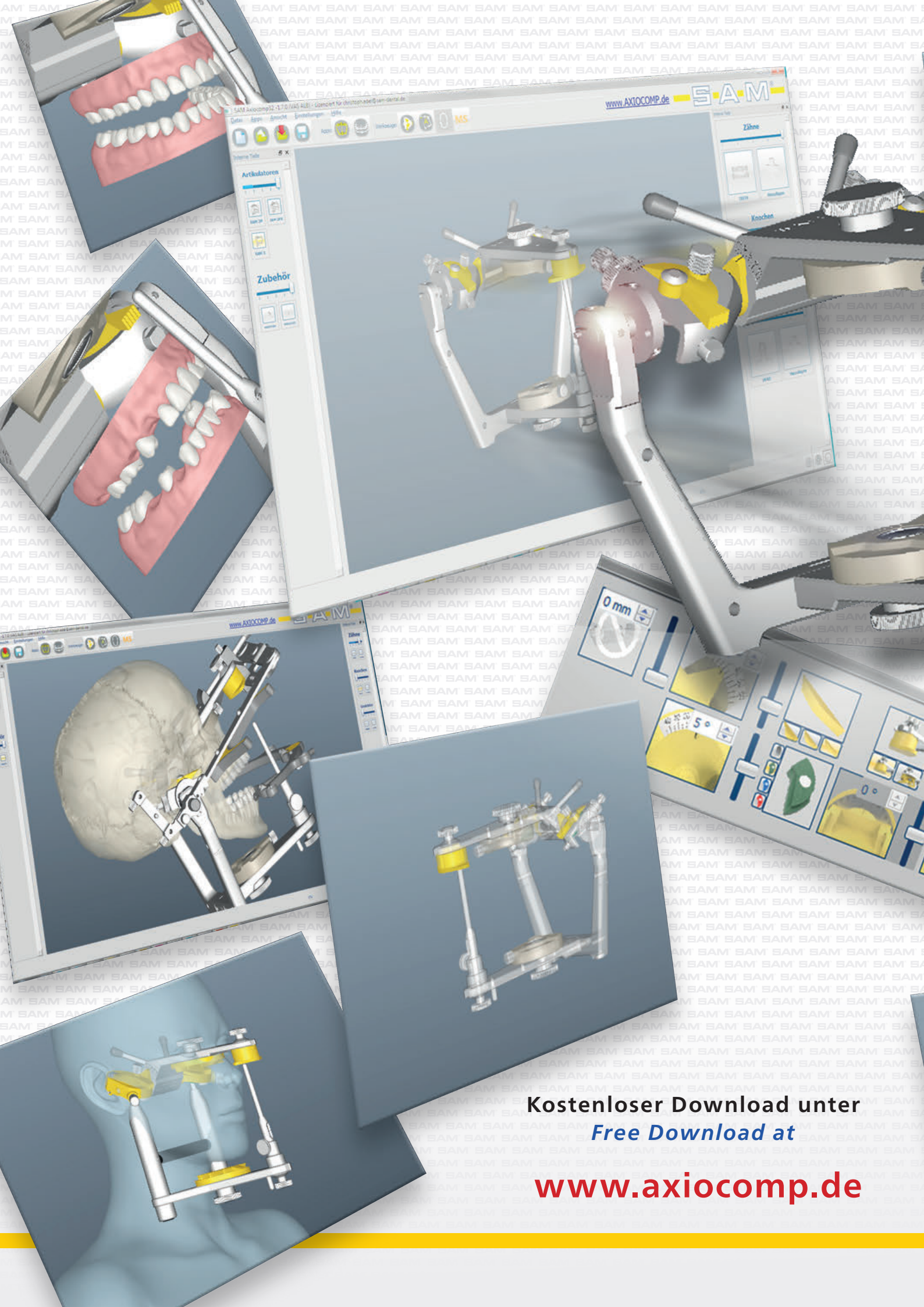
Due to the interchangeability of the SAM® system all transfer fork assemblies (ATB 205, ATB 210, ATB 220, ATB 305, ATB 310, ATB 320, ATB 395) can be used in the transfer stand ATB 398 and in all mounting instruments and articulators.

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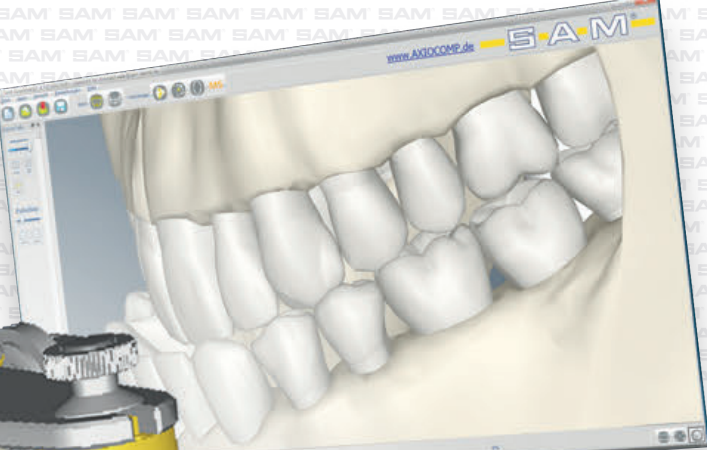
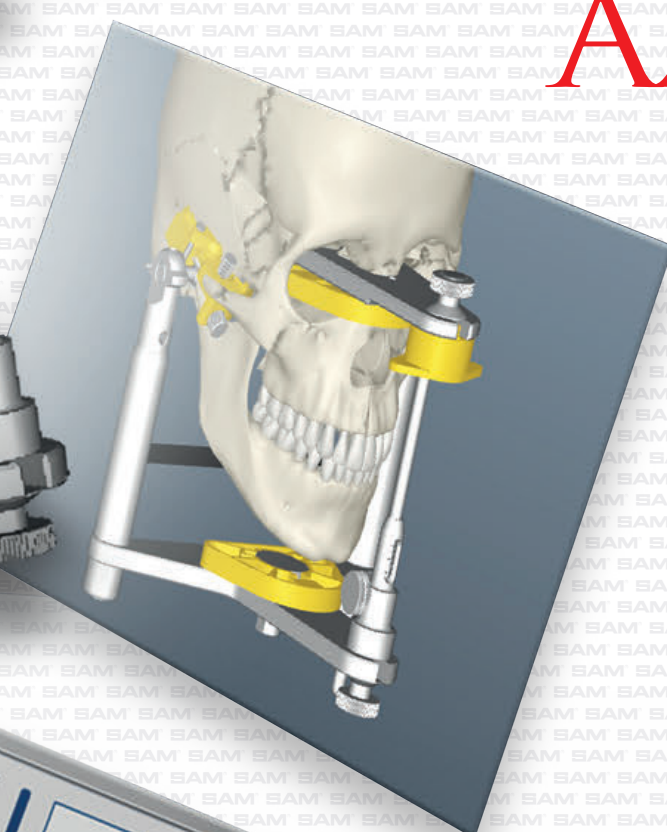
*Indispensable in teaching
and didactics!!*

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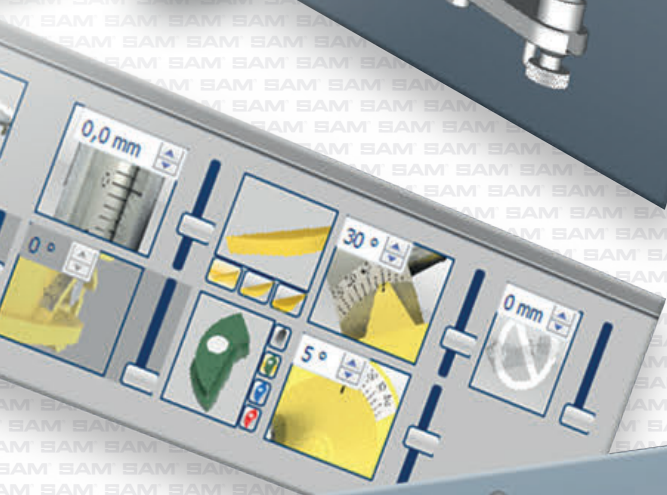
*Like your articulator,
is fully adjustable to handle
planning from laboratory to practice.*

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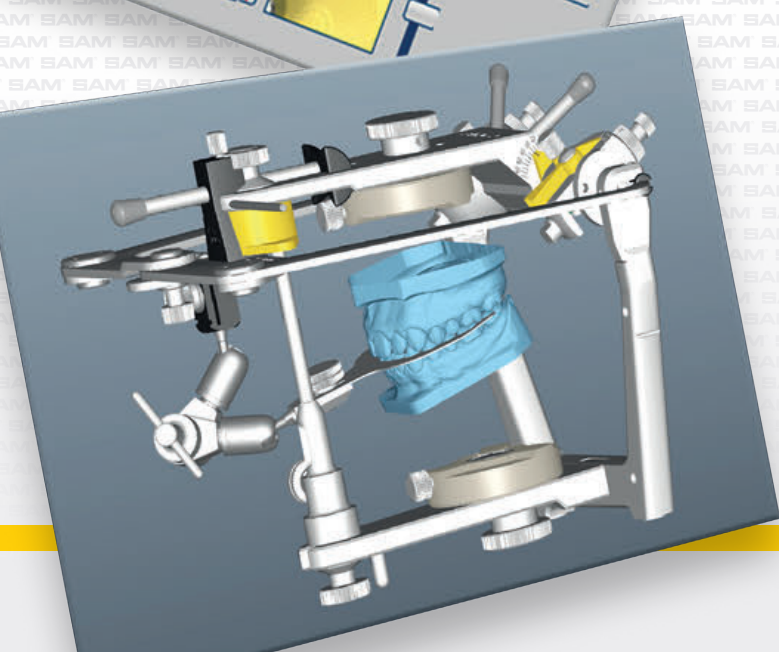*Many 3D-Views
and additional features*

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All of our products are all manufactured in Germany.

 Made in Germany

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Fussbergstrasse 1 • 82131 Gauting • Germany • Tel: +49 (89) 800 654 - 0 • Fax: +49 (89) 800 654 - 32
Email: info@sam-dental.de

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SAM® Präzisionstechnik GmbH ist DIN EN ISO 9001:2012 zertifiziert.
SAM® Präzisionstechnik GmbH is DIN EN ISO 9001:2012 certified.
SAM® Präzisionstechnik GmbH