## ART 530

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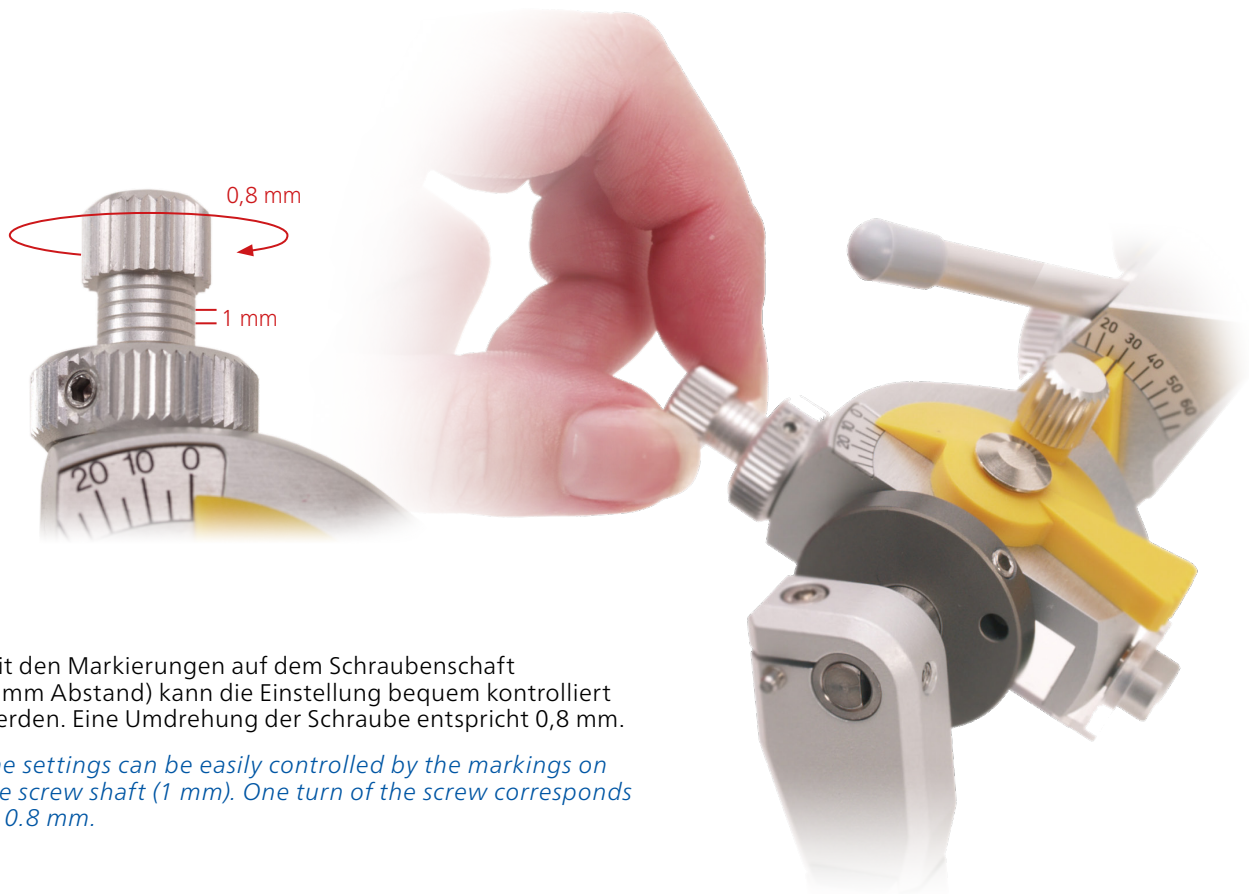
*EXCHANGE MANUAL  
PROTRUSION / RETRUSION SCREWS*

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*The settings can be easily controlled by the markings on the screw shaft (1 mm). One turn of the screw corresponds to 0.8 mm.*

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*For a straight protrusion, we recommend the straight Bennett Insert. A protrusive movement is also possible with the other inserts, when the small knurled nut is turned counterclockwise.*

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*For a retrusive movement, the large knurled nut of the protrusion screw is turned counterclockwise. One turn of the big screw is about 1mm.*

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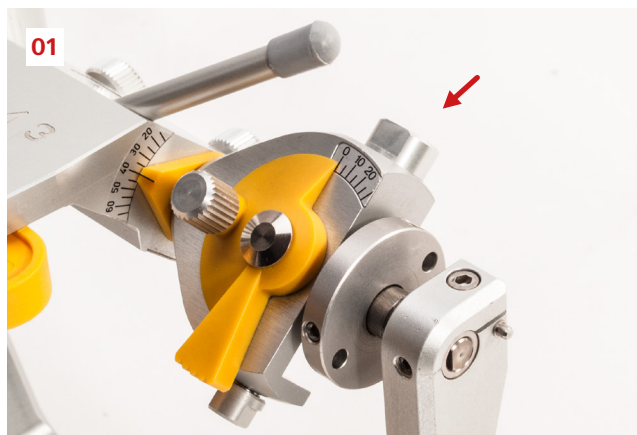
### For the installation and removal of ART 530


Protrusion / Retrusion screw in the SAM 3 the following items are needed:

- 1 Protrusions-/ Retrusions-Schrauben (ART 530)  
*Protrusion / Retrusion screw (ART 530)*
- 2 Inbusschlüssel 1,5 mm (ART 290001)  
*Hex wrench 1,5 mm (ART 290001)*
- 3 Steckschlüssel gelb (ART 510)  
*Yellow wrench (ART 510)*
- 4 Fühlerlehre 0,05 mm (z. B. in ART 187)  
*Feeler gauge 0,05 mm (f.e. contained in ART 187)*
- 5 Benneteinsatz gerade metall (ART 539)  
*Bennett Inserts straight metal (ART 539)*

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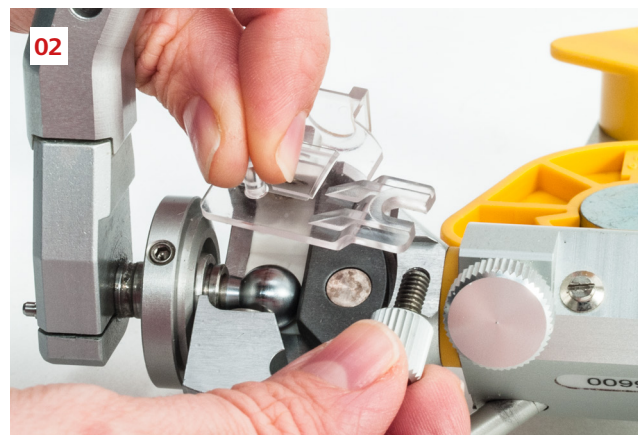*The SAM 3 Basic Articulator comes with an anvil (ART 532) on the top of the left and right condylar housing. This is replaced with a Protrusion / retrusion screw.*

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*To separate the upper member of the articulator from the lower, the transparent condylar housing covers have to be removed from the articulator.*

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*For replacement, the upper member of the articulator is separated from the lower.*

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*The anvil is loosened with the yellow wrench.*

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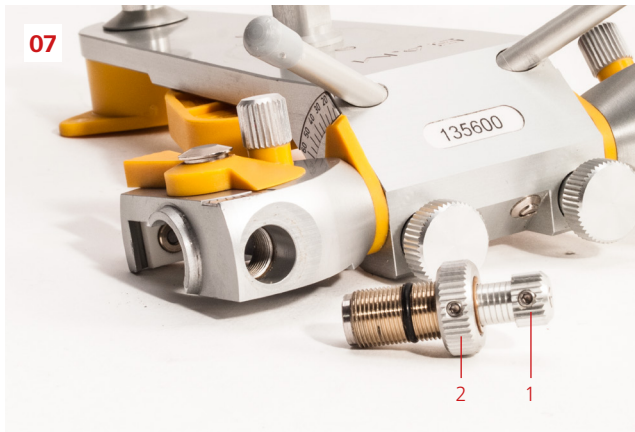
*The protrusion screw is then screwed into the released threaded bushing.*

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
*The protrusion screw is retractable and extendable. Before installation, the flat inner surface of the screw assembly **must be completely retracted** into the screw housing, there must be no gap at the lower end of the thread.*

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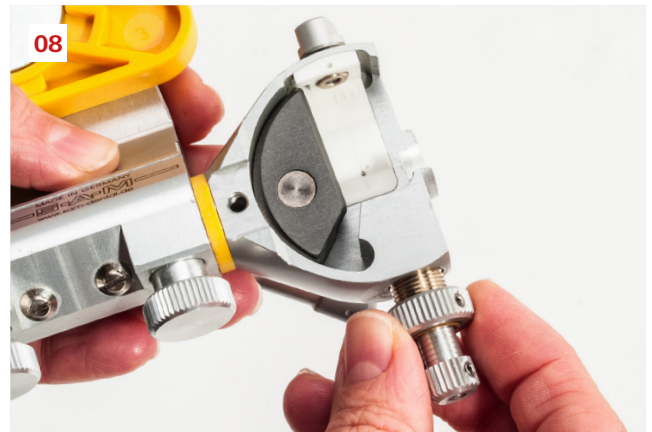*The large knurled nut (2) is rotated counter clockwise for retrusion, whereas, the small knurled nut (1) is used for protrusion.*

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
*The protrusion screw is screwed into the condylar housing assembly until the thread is fully seated at the stop.*

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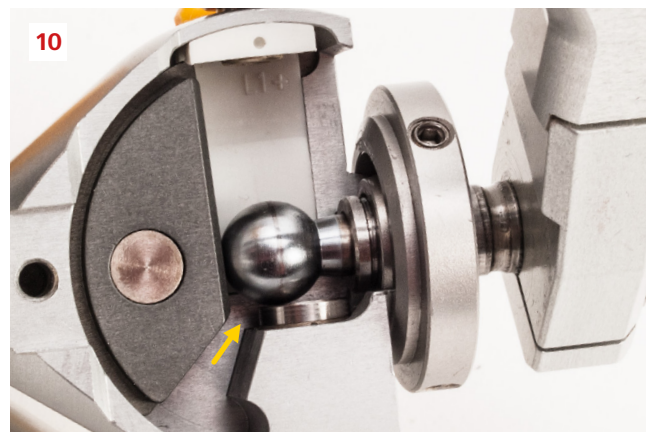*The upper member is resealed on the condylar elements of the articulator.*

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*After installation or adjustment, the distance between the ball and the screw head must always be checked.*

*For this, the centric locking device must be closed.*

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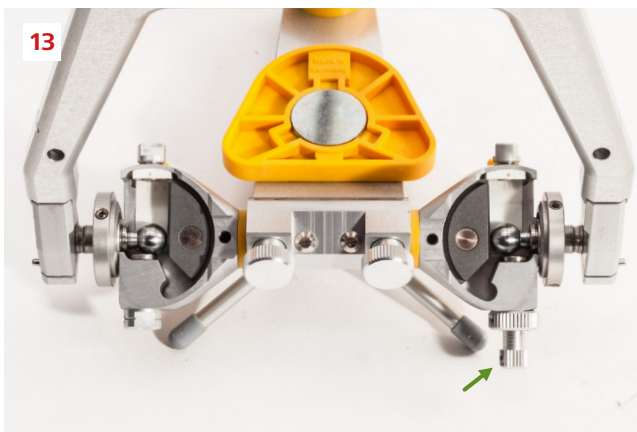
*With a 0.05 mm feeler gauge, the distance between the ball stop and the condyle ball is checked, which must be less than 0.05 mm. If this is not the case or if the centric lock cannot be closed, the protrusion screw must be readjusted.*

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
*Each time the protrusion screw is adjusted, the Bennett angle must be set to 0 ° and the condyle inclination to 40 °.*

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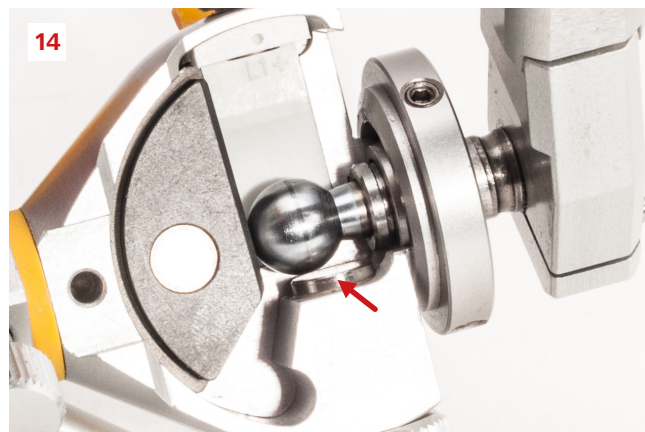*For a better view into the housing, the articulator should be turned over. Installation and adjustment of the protrusion screws should be done one at a time and not at the same time and be sure the both Centric Locking Rings are fully seated.*

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

#### Case 1:

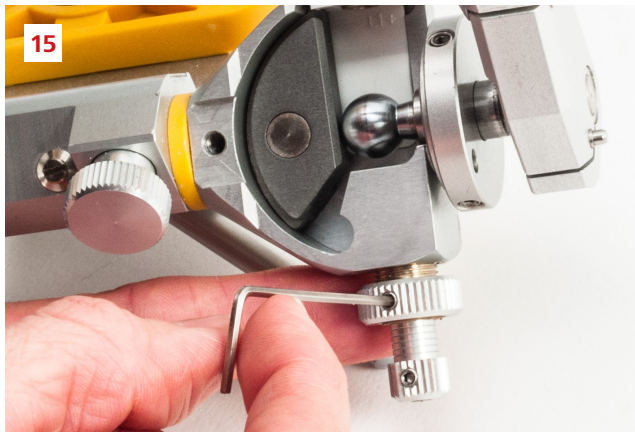*If there is a gap between the ball and the protrusion screw after installing the screw, **this has to be adjusted**. The Centric Locking Devices must be closed.*

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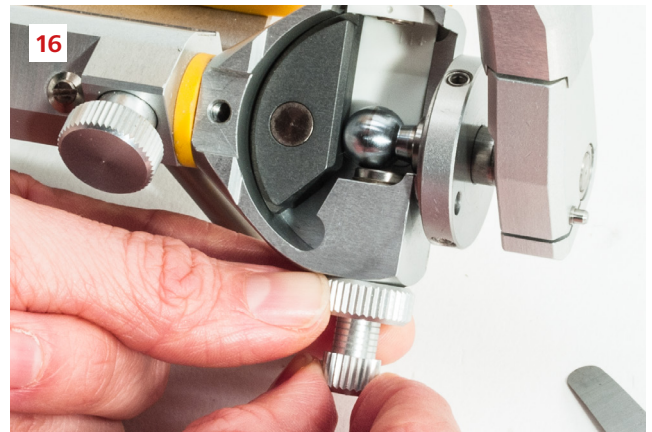
*The set screw is turned outwards a little bit from the housing and the retrusion screw is opened slightly with the Allen Wrench.*

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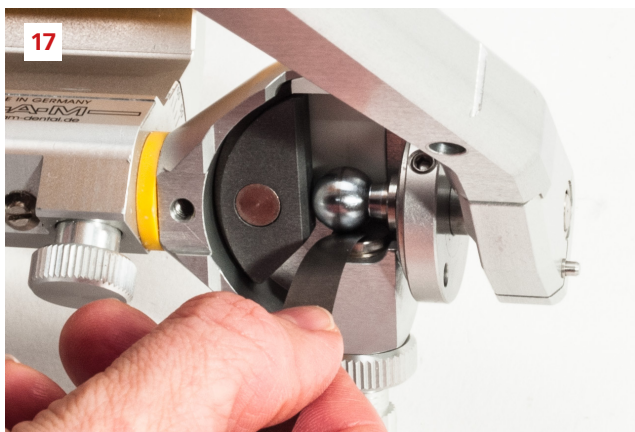*Next, only the large knurled screw is turned clockwise in the direction of the condyle ball; just before the stop, both knurled nuts must be tightened in conjunction with the feeling of contact of the screw assembly.*

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*With a feeler gauge, the distance is checked again (see 10-13). If the test reveals any defects, it is absolutely necessary to repeat the entire adjustment!*

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*If the distance between the screw head and ball fits, the Allen screw of the retrusion screw is tightened again in this position. For adjustment only the big knurled screw is used.*

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

**Case 2:**

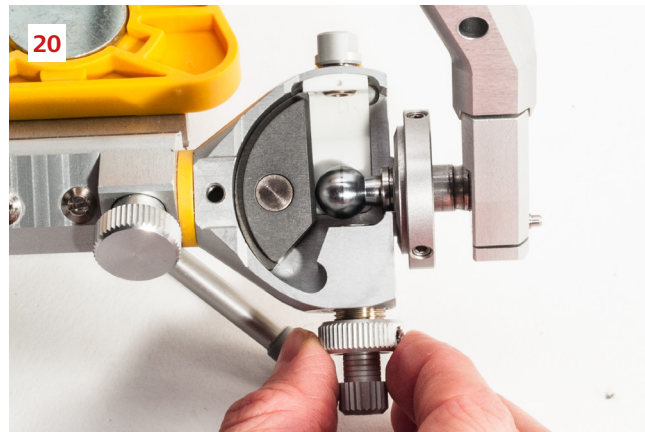*If after installing the protrusion- retraction screws, the centric locking device does not seat into position when closing, an adjustment must be made.*

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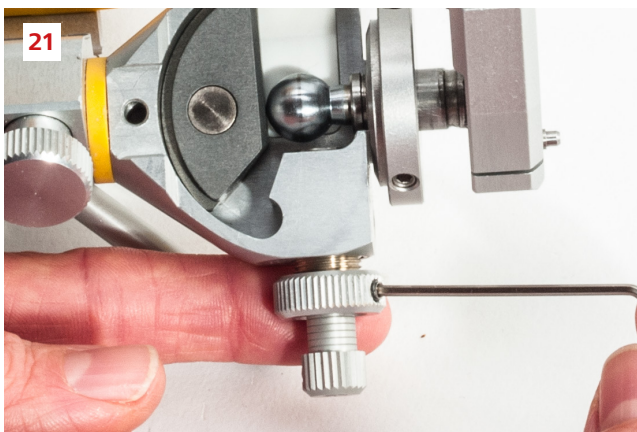
*First, the protrusion-retraction screw is turned out of the housing to the left until the centric lock can be closed without any obstruction.*

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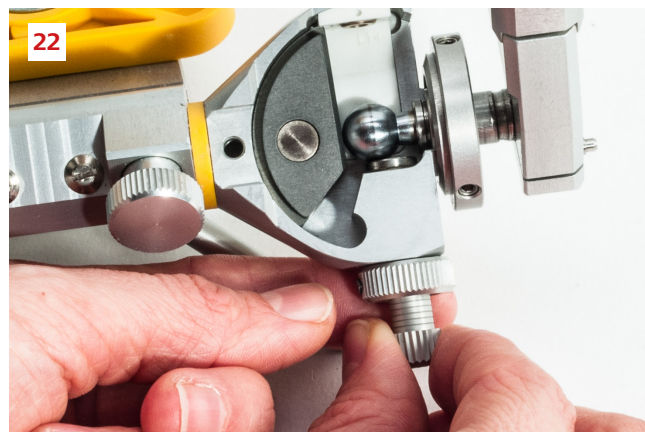*The retraction screw is opened slightly with the Allen wrench and screwed back.*

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*The small protrusion screw is turned clockwise towards the condyle ball. Shortly before the stop of the protrusion screw on the condyle ball, the retraction knurled nut is also turned clockwise so that the entire screw makes contact with the condyle ball.*

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
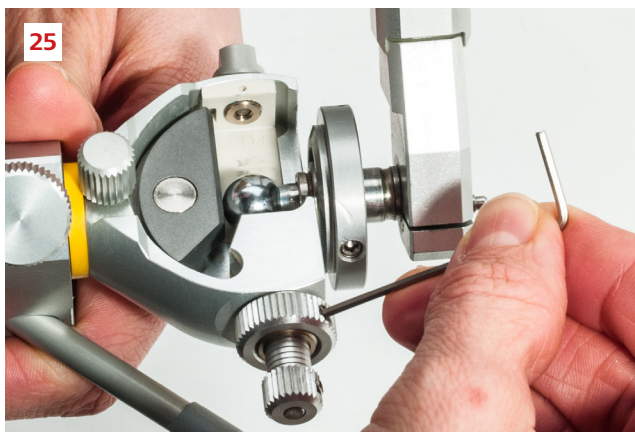
*If the distance fits, the Allen screw of the retrusion nut is tightened again in this position.*

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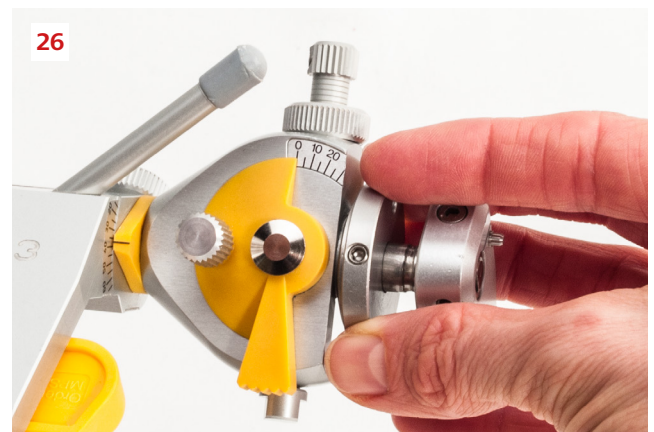
**Tip:**  
*If the screw is too tight, the Allen key can also be used as a lever to release the tension.*

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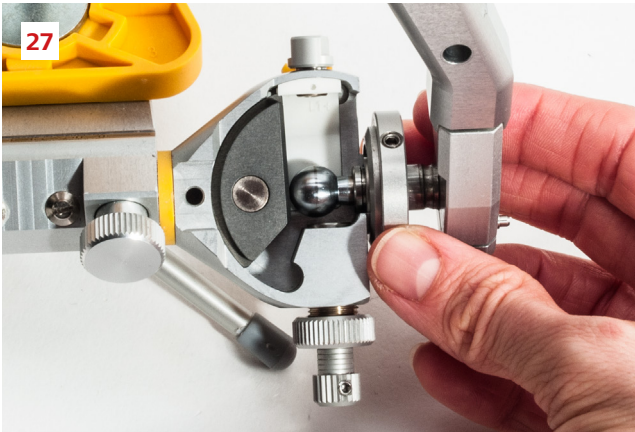**Control:**  
*When the centric lock is closed, the cone ring must be easy to turn.*

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*If this isn't the case, the screw must be brought into retrusion, which creates space in the housing. If the cone ring can now be rotated slightly, the screw is in a misalignment and the adjustment must be repeated again.*

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**Control:**

*The distance between Bennett guide and condyle ball must be less than 0.05 mm. This is to be checked with a feeler gauge.*

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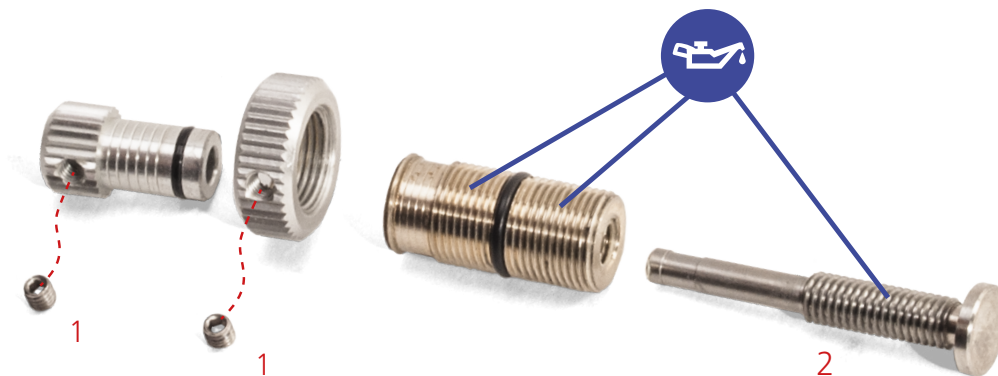
*If the protrusive movement on a straight Bennette insert angle 0° can not be carried out smoothly and the Bennett Guide prevents the condylar ball from a free movement, then this is an indication that the articulator should be sent to SAM for review.*

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### Tip:

*You should clean the Protrusion- Retrusion Screws after frequent use.*

*The small set screws 1 are loosened with the Allen Wrench and the screw is disassembled into its individual parts. Lubricate the screw sparingly with silicone grease on the threads (arrow).*

### During the subsequent reassembly please note:

1. *The illustrated sequence must be observed*
2. *The stopper bolt (2) must be screwed in counterclockwise.*
3. *Tighten the set screws only hand-tight!*

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*If any questions or problems arise during installation, SAM Customer Service can be contacted during our normal business hours.*

Tel: +49 89 800 654- 0  
Email: [service@sam-dental.de](mailto:service@sam-dental.de)

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SAM® Präzisionstechnik GmbH ist
*SAM® Präzisionstechnik GmbH is