**SPLITCAST-
ARTIKULATOR
PRÜFSOCKEL**

ART 396

***SPLITCAST
ARTICULATOR
VERIFICATION
DEVICE***

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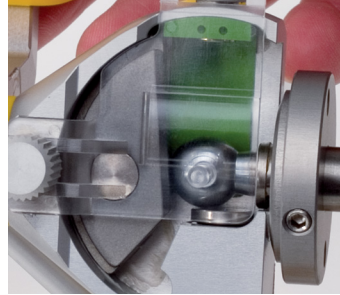
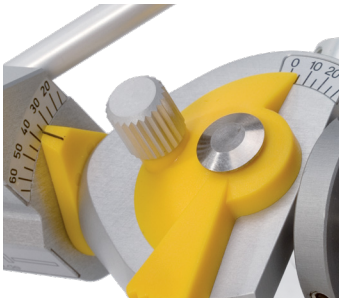
The Splitcast Articulator Verification Device ART 396 records the exact 3D distance between the mounting plates of any articulator and is used as a measuring gauge to assure articulator accuracy. It can also be used to control the accuracy of two or more factory synchronized identical articulators either in the dental office or laboratory.

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Put the straight Bennett insert in the articulator, set the Bennett-Scale to 5° and the Condyle Inclination Angle to 35°. Set the Incisal pin to 0.

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Set the Mounting Plates in position on the upper and lower part of the articulator. Place a acrylic block on top of the attached lower mounting plate to act as a space holder.

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Attach white rubber ring to exposed split cast side of device as shown in picture.

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Position the base of the verification device on the space holder centrally aligned such that the rubber ring is under the upper mounting ring.

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Make a soft creamy mix of Mounting Stone using 15 cc of water. Fill the rubbing ring and mounting plate with with the mixed stone.

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Close the articulator and carefully remove the excess mounting stone. Check the position again.

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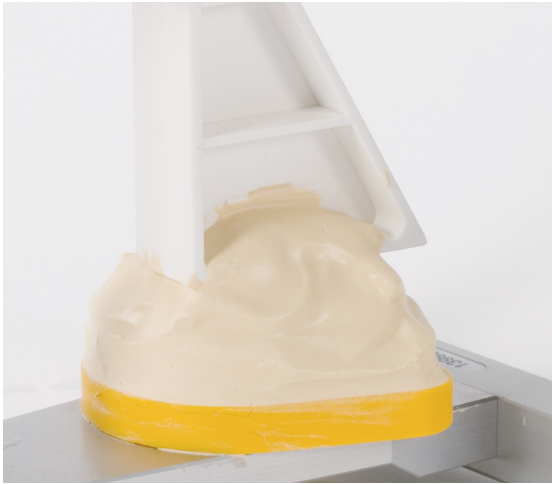
Once the mounting stone has set on the upper part of the verification, remove the mounting ring place holder from the lower part of the articulator. Make a soft creamy mix of Mounting Stone using 25 cc of water. Fill the lower mounting plate to a level such that it will contact and hold the lower part of the verification device.

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Close the articulator carefully into position such that the verification device bottom part contacts and attached to the mounting stone. Verify that Incisal Pin is in contact with the Incisal Table.

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It is recommended that the height of the mounting stone on the lower mounting plate be at least up to the level of the horizontal line on the lower part of the verification device.

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Allow dental stone to completely set and harden before opening the articulator and removing rubber ring.

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The completed splitcast articulator verification device.

SAM Präzisionstechnik GmbH
Fussbergstrasse 1 · 82131 Gauting · Germany · Tel: +49 (89) 800 654 - 0 · Fax: +49 (89) 800 654 - 32
Email: info@sam-dental.de